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FN 6852/01 | Duty room display with voice output

Original operating instructions

Getting started

What do these operating instructions cover?

These operating instructions apply to the following device:

FN 6852/01 Duty room display with voice output

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Distinctive feature: Nameplate ("x" = any digit)

Location of the nameplate:

On the motherboard
The nameplate is visible when the device is open. See "Installing the device" (Step 2), page 3.

Checking the scope of delivery

Please check that the scope of delivery is complete and in good condition. If any parts are missing or damaged, do not connect the device or put it into operation! File a complaint with the supplier.

The scope of delivery consists of:

Duty room display with voice output

Disposing of packaging

Dispose of the packaging in accordance with local regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions thoroughly. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk prevention.

Signal word	Meaning
GEFAHR	Warns of imminent threat to life or danger of serious injury.
WARNUNG	Warns of a potentially imminent threat to life or danger of serious injury.
VORSICHT	Warns of potentially imminent danger of minor or slight injury.
ACHTUNG	Warns of possible property damage.
HINWEIS	Indicates additional information.

Device information


Intended use

The device is used in duty rooms and ward offices.
Purpose:

- Recognizing and read calls
- Triggering calls
- Setting and resetting staff attendance
- Establishing voice connections

Any other use is improper and may result in malfunctions and damage to this device and connected devices. Failure to follow these operating instructions is also considered improper use.

Arrangement in the call system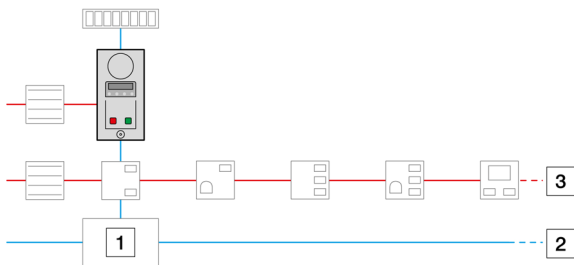


1 Central technology 2 System bus 3 Room bus

Features and characteristics


- Enclosure for surface-mounted installation over a flush-mounted switch box
- Housing with an antimicrobial film coating on the controls, easy to clean
- Audible call notification via ringtone
- Audible call forwarding
- Configurable features via call system management software from version 7.x via remote configuration
- Firmware update possible during operation via call system management software from version 7.x
- Data transmission and power supply via the FN 6000® system bus
- Connection to the system bus for the IP line module via a plug-in connector
- Connection to the room bus II to the call units via a terminal block

Controls and displays

The device has the following controls and indicators: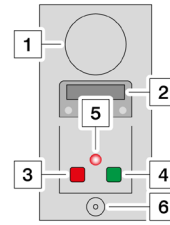

- 1 Speaker
- 2 Display
- 3 Call button
- 4 Cancellation button
- 5 Indicator light for voice connection (visible only when active)
- 6 Microphone

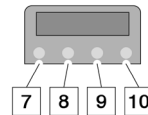- 7 "Up" arrow key
- 8 "Down" arrow key
- 9 Reset button for ringtone and call connection
- 10 Voice function button

- 11 Reassurance light (LED)
- 12 Button with orientation light (low-level button light)

Technical specifications

Physical data

Dimensions (W x H x D)	120 x 230 x 33 mm
Color	Pure white, similar to RAL 9010

Electrical data

Supply voltage	18–28 V DC
Standby power consumption	max. 31 mA
Current consumption at full load	max. 222 mA
Output current, room bus II	max. 750 mA, fused
Output current, signal lights (open-collector outputs)	max. 100 mA
Protection class	IP 40

Ambient conditions

Permissible ambient temperature	0...+40°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device must not be used in potentially explosive atmospheres.

 The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information

- Planning and testing of the call system, as well as certification of its functionality, must be carried out by a "specialist planner for call systems."
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is ensured by system separation.


Instructions for technical personnel

Ensuring staff qualifications

Ensure that all work is performed by suitable personnel:

- Installation and connection:
Qualified electrician or trained assistant
- Commissioning: "Call system specialist"

Procuring necessary accessories

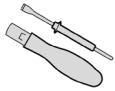For the connection terminals:

Torque screwdriver
Blade: max. 2.6 x 0.6 mm
Tightening torque: max. 0.2 Nm

If no device or connection box is installed:

Anchors Design suitable for the wall
Screws Half-round head, Ø max. 3.5 mm

Checking installation requirements

Before installation, ensure the following requirements are met:

- The system bus is designed as IY(St)Y 6 x 2 x 0.8.
- Room bus II is designed as IY(St)Y 2 x 2 x 0.6 (without voice) or as IY(St)Y 4 x 2 x 0.6 (for bed-by-bed voice communication)
- All cables to be connected are de-energized.
- The wall surface is flat and suitable for anchor fastening.


or:

- A flush-mounted switch box is installed: Ø 60 mm; depth min. 70 mm (a deeper switch box may make installation easier)

Do not install the device until all installation requirements have been met.

Documenting the serial number

To simplify documentation, each device has a two-part sticker with the serial number.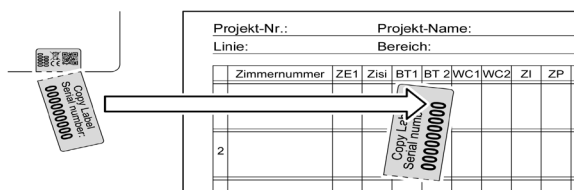

- Obtain the serial number list from the system administrator.
- Cut off the tag with the serial number.
Leave the part with the QR code on the device.
- Stick the tag with the serial number in the corresponding column. Examples:
ZE1 = Room electronics 1
Zisi = Room signal light
BT1 = Bed 1
WC1 = Toilet 1
WC2 = Toilet 2
ZI = Room
ZP = Additional circuit board
- If there are several identical devices per room:
Set up additional columns and number the columns consecutively. Example: "ZI 1", "ZI 2", etc.

Installing the device


For anchor installation:

Pipes or cables may run through plastered and clad walls.

When drilling, there is a risk of electric shock or danger from escaping gas or water.


Locate the drilling points using a cable detector.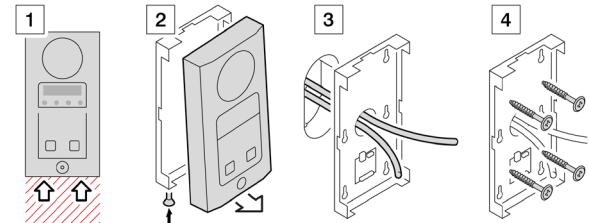

- When choosing the installation location, please note that the two housing screws must remain accessible.
- Loosen both screws at the bottom of the housing.
Tilt the housing away at the bottom.
This disconnects the plug-in connector.
Lift off the housing.
- Feed the cables through the bottom of the housing.
- Install the bottom of the housing.

Connecting the device

Basic information on connecting to a bus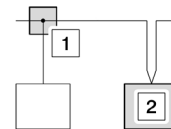

There are two options for connecting to a bus:

- In a bus junction box
- Looped through the device to be connected

Basic information on handling screw terminals


Applying force may damage the small screw terminal.


Use a torque screwdriver with a maximum torque of 0.2 Nm.

Gaining an overview

The device has the following ports:

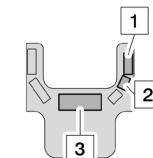
- Room bus II with audio
- Room bus II without audio
- System bus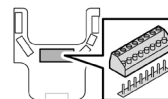


The system bus connection is pluggable.

Preparing the cables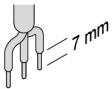


Strip the cable ends and insulate the wires.

Connecting room bus II with audio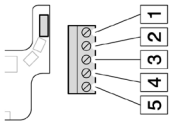


- 1 V_{PWR} (supply voltage)
- 2 GND (ground)
- 3 $Data_{LB}$ (room bus II data)
- 4 Audio 1
- 5 Audio 2

Connecting room bus II without audio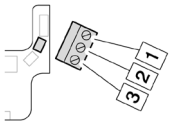


- 1 V_{PWR} (supply voltage)
- 2 GND (ground)
- 3 $Data_{LB}$ (room bus II data)

Connecting the system bus

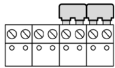
Leave jumpers 1-2 and 3-4 in their ports.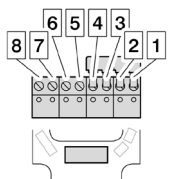

- 8 Audio 2
- 7 Audio 1
- 6 $Data_{SYST-BUS}$ (system bus data 2)
- 5 $Data_{SYST-BUS}$ (system bus data 1)
- 4 GND (ground)
- 3 GND (ground)
- 2 V_{PWR} (supply voltage)
- 1 V_{PWR} (supply voltage)

Closing the housing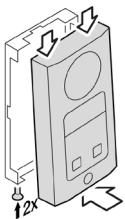

Hook the housing in at the top and swing it down against the bottom of the housing.

Press in firmly to make the internal connection.

Tighten both screws at the bottom of the housing.

Checking commissioning requirements

Before commissioning, ensure that the following requirements are met:

- The device is connected to the call system.
- The call system is active.

Only start up the device once all commissioning requirements have been met.

Starting up the device

Switch on the supply voltage. The device is now ready for use. Feature: The buttons light up (orientation light).

Notify the system administrator so that they can integrate the device into the call system from the call center.

Configuring the device

Configurations are not made on the device.

Notify the system administrator so that they can configure the device from the call center.

Keeping the device in working order

Performing quarterly inspections

Check for proper condition and function:

Signal lights and audible alarms

Performing annual inspection

Check for proper condition and function:

All other display devices

All call query controls (arrow buttons)

Performing annual maintenance

Perform the following maintenance tasks:

Maintain plant components

Repair

If unacceptable deviations from the target state of the call system are detected:

Perform the repair within 24 hours.

Record the repair in the operating log.

If the call system is completely or partially shut down for repair purposes:

Ensure that the affected rooms are monitored in some other way until the call system or the switched-off section is switched back on.

Repairing the device

ACHTUNG

The device performs safety-related tasks.

Improper repair may result in malfunction.

Only allow repairs to be carried out by the manufacturer.

Instructions for operating personnel


About the device's functions

This chapter describes the standard functions of the device. These functions can be changed in the central technology.

If your device behaves differently than described here, ask your system administrator to explain the changed functions to you.

Using the device

Detecting standby mode

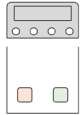All buttons are lit dimly (orientation light).


The display shows the time.

HINWEIS

The following instructions apply when attendance is set.

Recognizing an incoming call


The ringtone sounds every 10 seconds.
The display shows the call details.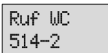

Line 1: Type of call (e.g., call from a WC)
Line 2: Source of the call (e.g., call from room 514, WC 2)

Recognizing an incoming emergency call

The ringtone sounds every second.
The display shows the emergency call details.

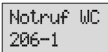Line 1: Type of call (e.g., emergency call from WC)
Line 2: Source of the call (e.g., emergency call from room 206, WC 1)

Recognizing an incoming voice call

Calls from voice terminals are indicated by the speaker icon.

Muting the ringtone

When a call is pending, you can mute the ringtone:

Press the reset button.
The ringtone stops.
The display remains on.


When a new call comes in, the ringtone is automatically turned back on.

Viewing the call list

If there are multiple calls pending:

The display shows all pending calls one after another in a continuous loop.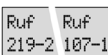

Higher-priority calls are displayed in alternating order with the other calls.

Scrolling through the call list

If there are multiple calls pending:

Use the "up" and "down" arrow keys to scroll through the call list.

6 seconds after the last time a scroll button is pressed, the display returns to the endless loop.

Triggering a call

Press the call button.

The call is triggered.

The reassurance light is on.

Triggering an emergency call

If staff attendance is set, press the call button as well.

The emergency call is triggered.

The red reassurance light flashes.

Acknowledging a call and setting staff attendance

Press the attendance button.

The call is acknowledged.

Staff attendance is set.

The reassurance light is on.

Resetting staff attendance

Press the attendance button again.

Staff attendance is reset.

The reassurance light goes out.

Establishing a voice connection

When a voice call is pending:

Press the scroll button to select the call. .


If there are multiple voice calls pending:

Use the scroll buttons to scroll through the call list until the desired call is selected.

Once the call is selected:

Briefly press the talk button.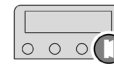

An audible signal sounds.

The red indicator light is on.

The voice connection is established.


To speak:

Press and hold the talk button.

To listen:

Release the talk button.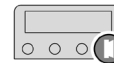

Ending a voice connection

Briefly press the reset button.

The call is disconnected.

The red indicator light goes out.

The call continues.

Ending a voice connection and acknowledging the call

Press and hold the reset button for at least 5 seconds.

The voice connection is disconnected.

The red indicator light goes out.

The call is acknowledged.

Instructions for support staff

Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use aggressive or abrasive cleaning agents.

Storing the device when not in use

Store the device in its original packaging in a suitable storage location. For requirements regarding the storage location, see "Technical Data."

Disposing of the device

After the end of the service life:

Observe regional disposal regulations. Have the device recycled properly. Do not dispose of the device in household waste!