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
Original operating instructions


### First steps

#### What do these operating instructions apply to?

These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features: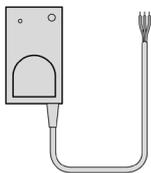

FN 6862/00 IMA diagnostic adapter

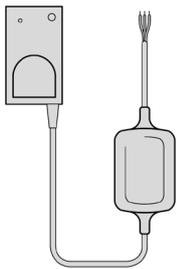FN 6862/01 IMA diagnostic adapter, 2 x MOPP

#### Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate:

On the back of the housing


On the back of the housing

#### Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of: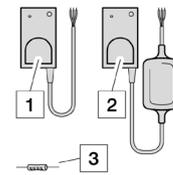

- 1 FN 6862/00 IMA diagnostic adapter or
- 2 FN 6862/01 IMA diagnostic adapter, 2 x MOPP
- 3 Resistor (10 kΩ) for cable break monitoring

#### What to do with the packaging?

If the device or parts of it are to be stored before or after use: Keep the packaging.

If nothing is to be stored:

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

#### Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

#### Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
<b>DANGER</b>	Warns of imminent danger to life or danger of serious injury.
<b>WARNING</b>	Warns of possible imminent danger to life or danger of serious injury.
<b>CAUTION</b>	Warns of possible imminent danger of minor or slight injuries.
<b>SAFETY INSTRUCTIONS</b>	Warns of possible material damage.
<b>NOTICE</b>	Indicates additional information.

## Information about the device

### Intended use

The device is connected to an IMA call unit and enables switching and signalling contacts of an external device to be connected to the call system.

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.


### Use of the FN 6862/00 IMA diagnostic adapter

- For connecting devices WITH safe system separation (2 x MOPP according to DIN EN 60601-1)
  - Alarm contact of an ECG device
  - Fill level warning of a perfusor
- For connecting PASSIVE devices
  - Footfall contact of a doormat

### Use of the FN 6862/01 IMA diagnostic adapter, 2 x MOPP

- For connecting mains-powered devices WITHOUT safe system isolation

### Arrangement in the call system

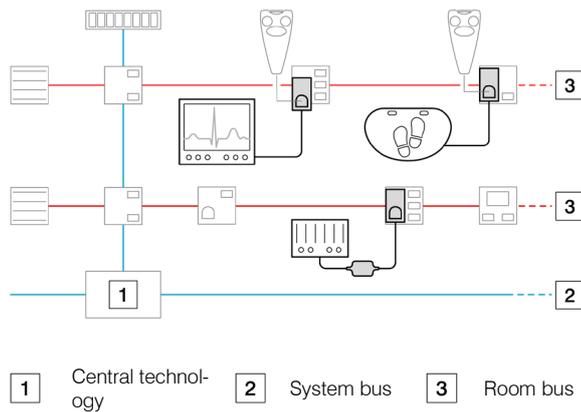### Features and properties

- Double-sided IMA (intelligent solenoid connection) for connecting, for example, an additional handheld button to the same call unit
- Two diagnostic adapters and e.g. an additional handheld button can be connected to the same call unit
- Suitable for all IMA call units
- Antimicrobial surface, easy to clean
- Configurable performance features
  - via call system management software from version 7.6
  - via remote configuration
- Firmware update possible during ongoing operation via call system management software from version 7.6
- Data transmission and power supply via the FN 6000® system bus

### Controls and displays

- 1 Log in and log out button (feature: small dot)
- 2 LED (red)
- 3 IMA connection (rear)
- 4 IMA connection (front)
- 5 Medical isolator (only for FN 6862/01 IMA diagnostic adapter, 2 x MOPP)
- 6 Connection cable to the external device

### Technical data

#### Physical data

	FN 6862/00	FN 6862/01
Dimensions		
Housing [mm] (W x H x D)	33 x 55 x 12	54 x 75 x 22
Total length incl. cable [m]	approx. 2.50	approx. 5.00
Weight incl. cable [g]	approx. 140	approx. 275
Colour	Pure white (similar to RAL 9010)	

#### Electrical data

Supply voltage	18...28 V DC
Contact inputs	Potential-free normally open or normally closed contact (no external voltage!) Fault signalling contact
Protection class	IP 30

#### Ambient conditions

Permissible ambient temperature	0...+40°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device must not be used in potentially explosive atmospheres.

**CE** The declaration of conformity can be requested from the manufacturer. See last page for contact details.

#### Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

## Instructions for technical staff

### Connecting the diagnostic adapter to the external device

Ensure that an external medical device is only connected by a specially trained professional.

#### Ensuring galvanic isolation

**⚠ DANGER** The diagnostic adapter has no galvanic isolation.

Ensure adequate protection on the external device side (2 x MOPP in accordance with DIN VDE 0834 2016:06 and DIN EN 60601).


#### Which contacts can be connected?

The contact outputs of the external device must have the following properties:


Contact type	Normally closed contact (NC) or normally open contact (NO)
Voltage	Potential-free switching contacts only No external voltage!
Safety	Only contacts from devices with safe system separation (2 x MOPP according to DIN EN 60601-1)
Cable break monitoring	Can be set up by connecting resistors (included in the scope of delivery).

**NOTICE** The functionality of the contact outputs must be configured individually for the connected external device in the call system management software.


#### Connection options for FN 6862/00 IMA diagnostic adapter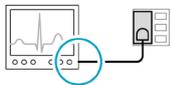


The following details show the connection of the IMA diagnostic adapter to the switching outputs of the external device.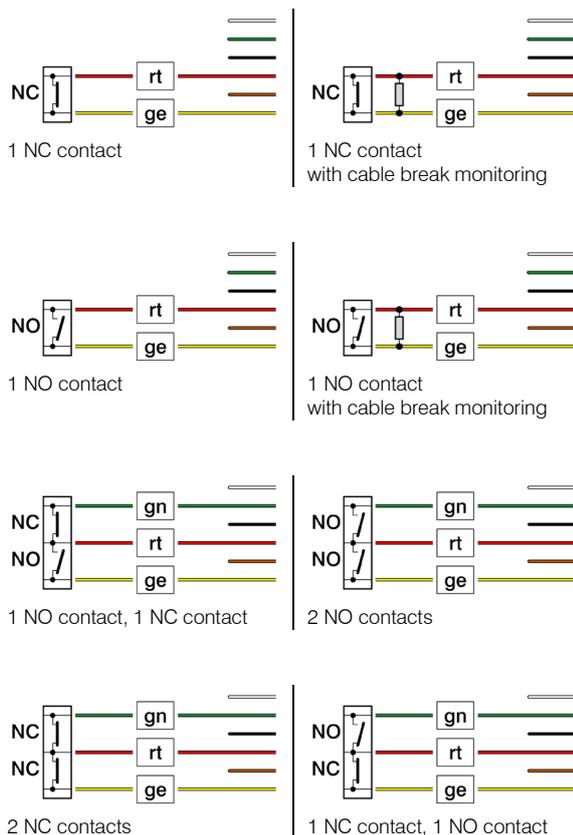

#### Connection options for FN 6862/01 IMA diagnostic adapter, 2 x MOPP

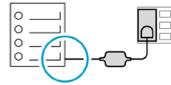The following details show the connection of the IMA diagnostic adapter to the switching outputs of the external device.

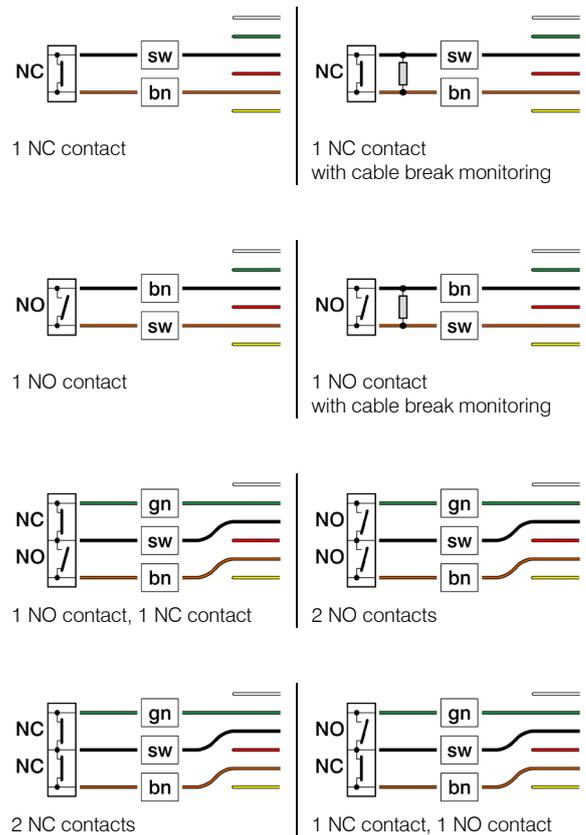## Configuring and commissioning the device

Ensure that the device is only put into operation by a "specialist for call systems".

Notify the system administrator so that they can configure the device from the call centre and integrate it into the call system. The device is then ready for use.

## Servicing and maintaining the device

Ensure that this device and the entire call system are fail-safe at all times. Follow the regulation DIN VDE 0834-1:2016-06 (11.2 to 11.4).

## Repairing the device

**SAFETY INSTRUCTIONS** The device fulfils safety-relevant tasks. **There is a risk of malfunction if the device is repaired incorrectly. Only have repairs carried out by the manufacturer.**

## Connecting the diagnostic adapter

- 1 Move the diagnostic adapter towards the socket on the call unit. The plug is magnetically attracted and snaps into the socket.

- 2 The red reassurance light on the call unit flashes quickly. The diagnostic adapter is recognised.

- After 3 seconds, the red reassurance light turns off. The diagnostic adapter is registered.

## Testing the function

- 1 If possible on the external device: Test the function of the diagnostic adapter by simulating a diagnostic call. (See the operating instructions for the external device).

- 2 When the call is triggered, the red reassurance light must light up on the call unit.

- 3 Reset the call. (See the operating instructions for the call unit.)

## Instructions for operating staff


### Connecting the diagnostic adapter

#### Which connection options are available?


The following components can be connected to a call unit:

- A diagnostic adapter for an external device
- An additional diagnostic adapter for another external device
- A handheld button


Examples: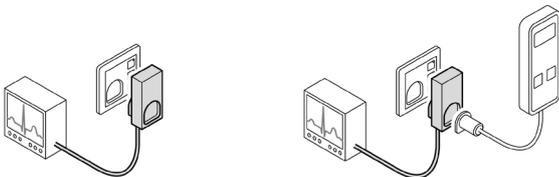


*A diagnostic adapter*

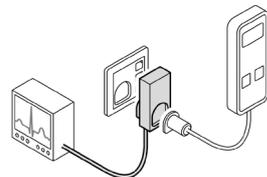*A diagnostic adapter and a handheld button*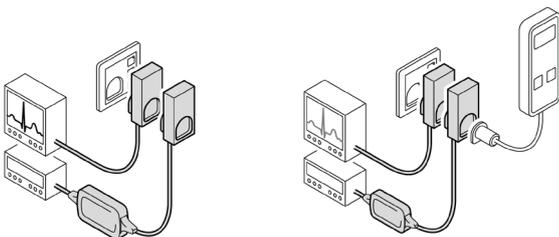


*Two diagnostic adapters*

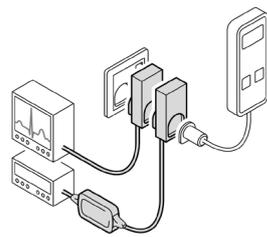*Two diagnostic adapters and a handheld button*

### Using the handheld button

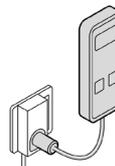
An additionally connected handheld button functions as usual – independently of the diagnostic adapter. (See the operating instructions for the handheld button.)

### Disconnecting the diagnostic adapter from the call unit

#### Temporarily disconnecting the diagnostic adapter

Case study: Tidying up the cable routing and reconnecting the diagnostic adapter.

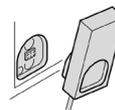- 1 Tilt the diagnostic adapter forwards at the top and remove it.

The red reassurance light flashes on the associated call unit.

- 2 Within 10 seconds: Reconnect the diagnostic adapter. Move the plug towards the socket on the call unit. The plug is magnetically attracted and snaps into the socket.

The reassurance light on the call unit turns off.

### Exceeded the 10-second time limit?

If you have exceeded the time limit of 10 seconds, the following condition has occurred:

- The call unit has triggered a tear-off alarm in the central technology.
- The tear-off report is shown on the display units (e.g. on the corridor displays and on the staff room displays)
- The red reassurance light continues to flash.
- The central technology has de-registered the diagnostic adapter.
- The diagnostic adapter must be re-registered.


To register the diagnostic adapter again, reconnect it.

After 3 seconds:

- The diagnostic adapter is registered again.
- The red reassurance light turns off.
- The tear-off report is deleted.

### Replacing the diagnostic adapter with another

Case study: Taking the diagnostic adapter for repair and connecting a replacement device.

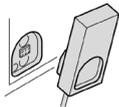- 1 Tilt the diagnostic adapter forwards at the top and remove it.

The red reassurance light flashes on the associated call unit.

After 10 seconds, the associated call unit triggers a tear-off report in the central technology.

- 2 Press and hold the green button on the associated call unit until the red reassurance light turns off.

The diagnostic adapter is de-registered.

### Connecting another diagnostic adapter

- 3 Move the plug of the other diagnostic adapter towards the socket.  
The plug is magnetically attracted and snaps into the socket.

The red reassurance light on the associated call unit flashes quickly.

The diagnostic adapter is recognised.

After 3 seconds, the red reassurance light turns off.

The diagnostic adapter is registered.


- 4 Check that the diagnostic adapter is working properly.

### De-registering the diagnostic adapter

Case study:

The diagnostic adapter is no longer to be used on this call unit.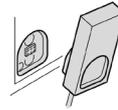

- 1 Tilt the diagnostic adapter forwards at the top and remove it.

The red reassurance light flashes on the associated call unit.

- 2 Press and hold the green button on the associated call unit until the red reassurance light turns off.

The diagnostic adapter is de-registered.

## Instructions for auxiliary staff

### Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

### Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

#### SAFETY INSTRUCTIONS

**The device contains heat-sensitive components.**

**There is a risk of destruction when exposed to heat.**

**Do not sterilise the device thermally.**

### Storing the device when not in use

Store the device in its original packaging in a suitable storage location. For the requirements for the storage location, see "Technical data".

### Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!