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Original operating instructions

First steps

What do these operating instructions apply to?

These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features:

FN 6743/23 Acoustic switch
with NK connection
(ancillary plug contact)

FN 6843/21 Acoustic switch
with IMA connection
(intelligent solenoid connection)

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate:


On the base of the housing

Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of: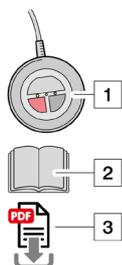

- 1 Acoustic switch
- 2 Operating instructions
- 3 Download the latest version of the operating instructions at <https://www.tetronik-kommunikationstechnik.de/downloads/original-betriebsanleitungen/>.
Search term: FN number of your device.

What to do with the packaging?

If the device or parts of it are to be stored before or after use: Keep the packaging.

If nothing is to be stored:

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
DANGER	Warns of imminent danger to life or danger of serious injury.
WARNING	Warns of possible imminent danger to life or danger of serious injury.
CAUTION	Warns of possible imminent danger of minor or slight injuries.
SAFETY INSTRUCTIONS	Warns of possible material damage.
NOTICE	Indicates additional information.

Information about the device


Intended use

The device is used in the resident's or patient's room. Purpose: Contactless triggering of a call by an acoustic signal, e.g. a concise sound or speech of the resident or patient.

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

⚠ WARNING The device may only be used as a support in the care field.
It is not suitable for checking on persons subject to mandatory monitoring.

Arrangement in the call system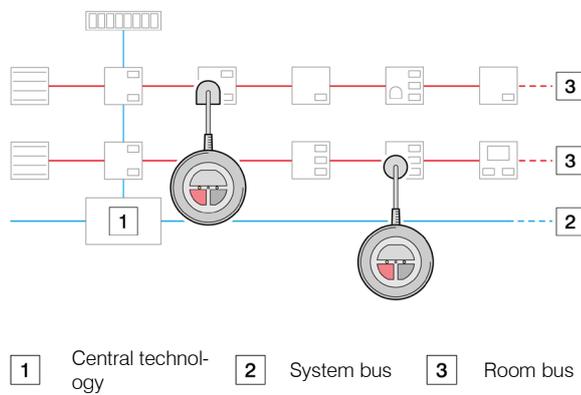


Features and properties

- Call triggering by
 - speech
 - concise sound
 - manual operation of the call button
- Adjustable microphone properties:
 - Sensitivity
 - Noise filter
 - Mute
- Temporary emergency call block with attendance set
- Connection to a room module in the room bus
- Data transmission and power supply via the room bus


Controls and displays

The device has the following controls and displays: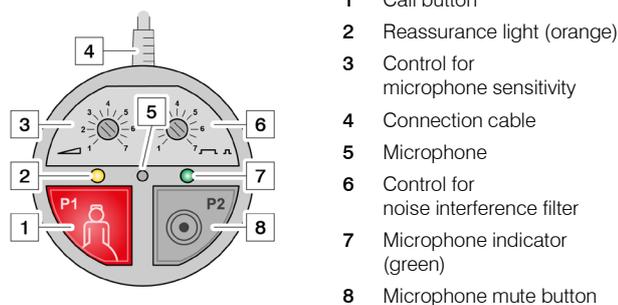

Operator connections

Depending on the model, the device has one of these operator connections: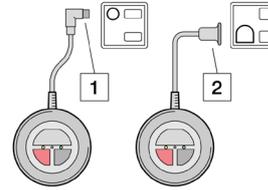

- 1 Plug for room modules with NK connection (auxiliary plug contact)
- 2 Plug for room modules with IMA connection (intelligent solenoid connection)

Technical data

Physical data

Dimensions (Ø x H)	110 x 50 mm
Cable length	2.65 m
Weight (with cable)	216 g

Electrical data

Supply voltage	+24 V DC (room bus)
Input current	min. 10 mA; max. 40 mA
Protection class	IP 30

Ambient conditions

Permissible ambient temperature	+10...+25°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device must not be used in potentially explosive atmospheres.

CE The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

Instructions for technical staff

Keeping the device functional

Ensure that the work described in this section is carried out by a "specialist for call systems"

Conducting a quarterly inspection

Check for perfect condition and function:

- Call buttons and movable call triggering devices intended for use by residents or patients
- Signal lights and acoustic signalling devices

Conducting an annual inspection

Check for perfect condition and function:

- All other devices for call triggering, call cancellation and attendance reporting
- All other display devices
- All connected devices for receiving, forwarding and displaying calls, e.g. plug-in devices for connecting call-triggering devices

Conducting annual maintenance

Carry out the following maintenance work:

- Maintain system components

Repair

If unauthorised deviations from the set status of the call system are detected:

- Carry out the repair within 24 hours.
- Record the repair in the operating log.

If the call system is completely or partially switched off for maintenance:

- Ensure that the affected rooms are otherwise checked until the call system or the switched-off part is switched on again.


Repairing the device

SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks. There is a risk of malfunction if the device is repaired incorrectly. Only have repairs carried out by the manufacturer.

Instructions for operating staff

Setting up the device

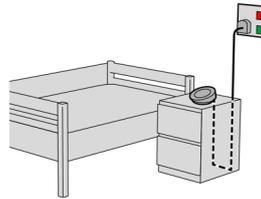Place the device on a level surface. Ensure that the installation location fulfils the following requirements:

- The device is within reach and calling distance of the resident or patient.
- The cable reaches as far as the room module when laid safely.

CAUTION Exposed cable. Danger of tripping. Do not lay cables in walkways.

Connecting the device

Device with NK connection:
Insert the plug into the NK socket of the room module.

Device with IMA connection:
Move the plug towards the socket on the associated call unit.
The plug is magnetically attracted and snaps into the socket.

The red reassurance light on the associated call unit flashes quickly.

The device is recognised.

After 3 seconds, the red reassurance light turns off.

The device is registered.

Configure the device now to adapt it to the local conditions.

Configuring the device

Setting the microphone sensitivity


The microphone sensitivity determines how loud a noise needs to be to trigger a call.

The aim of the setting is to achieve a balance between reliable call triggering and the probability of false triggering.

Guide values for setting the microphone sensitivity:

Microphone sensitivity	Level	Required volume	Frequency of false triggering
Low	1	Very loud	Rare
Medium	4	Medium	Occasional
High	7	Very quiet	Frequent

The appropriate setting depends on the situation and is determined by testing the device when it is ready for use:

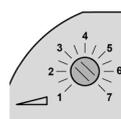Use the screwdriver to set the controller to a medium value.

Stand at an average distance from the device.

Speak at an increasing volume and observe the reassurance light.

As soon as the reassurance light lights up, a call has been triggered.

Cancel the call again on the room module.

If necessary, carry out further tests with other settings until you have found the right sensitivity.

Setting the noise interference filter

The digital noise interference filter disregards background noise to make it easier to recognise call-triggering noises.

The aim of the setting is to achieve a balance between reliable call triggering and the probability of false triggering.

Guide values for setting the noise interference filter:

Setting level 1 (low)

Effect: Loud background noises are disregarded (e.g. radio or TV). Significantly louder noises trigger a call (e.g. calling "Help!").

Error rate: Low

Setting level 3 (medium)

Effect: Moderate, constant background noise is disregarded. Significantly louder and longer-lasting noises trigger a call.

Error rate: Medium

Setting level 5 (high)

Effect: Almost any noise that stands out clearly from quieter and constant background noises triggers a call.

Error rate: High


Setting level 7 (maximum)

Effect: Hardly any background noise is disregarded. Any noise that occurs triggers a call.

Error rate: Very high

The appropriate setting depends on the situation and is determined by testing the device when it is ready for use:

Make the setting with the average background noise.

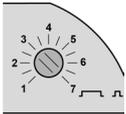Use the screwdriver to set the control to the desired level and observe the LEDs lighting up. The LEDs each light up for 1 second and then go out again:

Level 1

Level 3

Level 5

Level 7

Stand at an average distance from the device.

Make a noise or speak at the volume that should trigger a call.

As soon as the reassurance light lights up, a call has been triggered.

Cancel the call again on the room module.


If necessary, carry out further tests with other settings until you have found the right setting.

Activating a temporary emergency call block

The emergency call block prevents an emergency call from being triggered by another noise when the attendance is set.

The emergency call block is activated after the attendance is set and is effective for 20 minutes.

To activate the emergency call block:

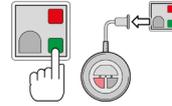Set the attendance.
Pull out the acoustic switch.

Press and hold the button P1.
Reconnect the acoustic switch.
Release the button P1.

The green LED flashes.

To activate or deactivate the emergency call block:

Press the P1 button multiple times to select:
LED on: Emergency call block on
LED off: Emergency call block off

To confirm the setting:
Press the button P2.

Switching off acoustic call triggering

If calls are only to be triggered using the call button, acoustic call triggering can be switched off. The microphone is muted for this purpose.

To switch off the acoustic call triggering:

If the LED is off:
The microphone is switched on.

To mute the microphone:
Press the P2 button.

The LED flashes for 10 seconds.


As soon as the LED lights up:
The microphone is muted.
To switch the microphone back on:
Repeat the process.

Using the device

Triggering a call


Triggering a call manually:
Press the call button.
The call is triggered.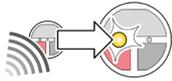

Triggering a call acoustically:
Speak loudly or make a noise.
The call is triggered.

Cancelling a call

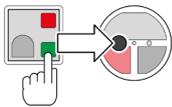A call can only be cancelled on the room module.
Press the cancellation button.
The LED turns off.
The call is cancelled.

Disconnecting the device

Device with NK connection:
Pull the plug out of the NK socket of the room module.

Device with IMA connection:
Tilt the plug to the side and remove it.

The red reassurance light flashes on the associated call unit.

Press and hold the green button on the associated call unit until the red reassurance light turns off.

The device is de-registered.

Teaching residents or patients

Explain all the functions that are important to the resident or patient:

- Triggering a call

Instructions for auxiliary staff

Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

Disinfecting the device

Disinfect the surface of the device with a commercially available surface disinfectant in accordance with EN 16615 and the instructions.

SAFETY INSTRUCTIONS

**Heat-sensitive components.
Danger of destruction.
Do not thermally sterilise the device.**

Storing the device when not in use

Store the device in its original packaging in a suitable storage location. For the requirements for the storage location, see "Technical data".

Disposing of the device

At the end of the service life:
Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!