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FN 671x/8x


Call and cancellation button vandal-proof, aluminium

Original operating instructions


First steps

What do these operating instructions apply to?


These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features: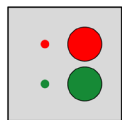

FN 6716/82
Call/cancellation button
Vandal-proof
Aluminium design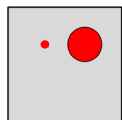

FN 6715/81
Call button
Vandal-proof
Aluminium design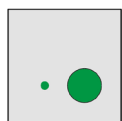


FN 6712/81
Cancellation button
Vandal-proof
Aluminium design

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate: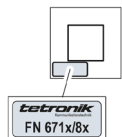


On the back of the front panel


Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of: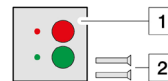

- 1 FN 6726/81 Call/cancellation button
- 2 2 x 2-hole safety screw

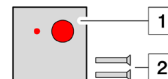- 1 FN 6725/81 Call button
- 2 2 x 2-hole safety screw

- 1 FN 6722/81 Cancellation button
- 2 2 x 2-hole safety screw

Disposing of packaging

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
DANGER	Warns of imminent danger to life or danger of serious injury.
WARNING	Warns of possible imminent danger to life or danger of serious injury.
CAUTION	Warns of possible imminent danger of minor or slight injuries.
SAFETY INSTRUCTIONS	Warns of possible material damage.
NOTICE	Indicates additional information.

Information about the device


Intended use

The device is used in areas at risk of vandalism. Purpose (depending on equipment):

- Triggering a call
- Cancelling a call and setting staff attendance
- Triggering an emergency call

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

Arrangement in the call system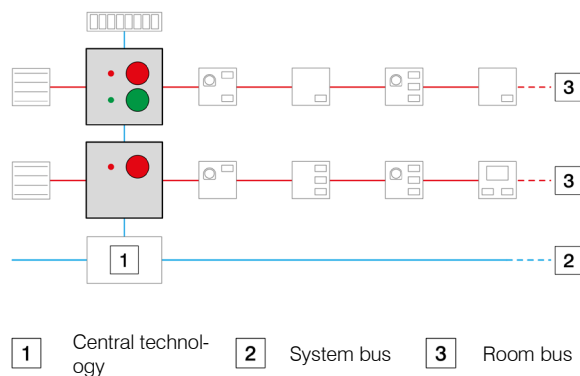

Features and properties

FN 6716/82 Call/cancellation button

- Call button (red)
- Orientation light/reassurance light (red)
- Cancellation button (green)
- Orientation light/reassurance light (green)

FN 6715/81 Call button

- Call button (red)
- Orientation light/reassurance light (red)

FN 6712/81 Cancellation button


- Cancellation button (green)
- Orientation light/reassurance light (green)

Similarities


- Vandal-proof aluminium design
- Data transmission and power supply via the FN 6000® system bus
- Connection to the room bus via terminal strip
- Monitoring of all call units installed in the room
- Connection of up to 8 beds and up to 8 WC call units to the room electronics
- Bed-by-bed call selection
- Separate call reset in the WC area, in accordance with DIN VDE 0834, programmable
- Configuration via programming device
- Orientation light function in accordance with DIN VDE 0834
- Buzzer for acoustic signalling of forwarded calls
- Flush-mounted design for installation in a switch box (Ø 60 mm; depth min. 50 mm)

Controls and displays


FN 6726/81 Call/cancellation button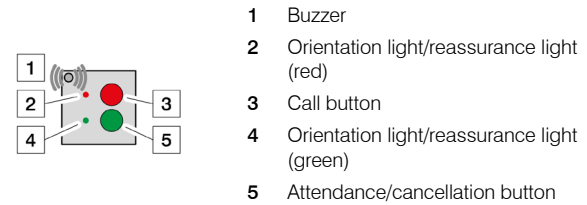

FN 6726/81 Call button

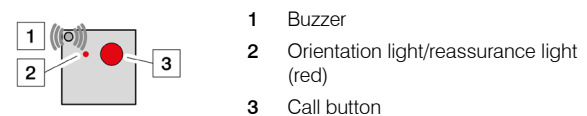FN 6722/81 Cancellation button

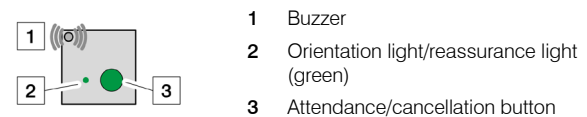Technical data

Physical data

Dimensions (W x H x D)	81 x 81 x 42 mm
Weight	approx. 100 g
Front panel material	Aluminium 6082

Electrical data

Supply voltage	18...28 V DC
Input current	<u>22 mA</u> (+ 10 mA per LED)
Protection class	IP 20

Ambient conditions

Permissible ambient temperature	+10...+40°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device must not be used in potentially explosive atmospheres.

CE The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.


Instructions for technical staff

Ensuring staff qualification


Ensure that all work is carried out by suitable staff:

- Installation and connection:
Qualified electrician or trained assistant
- Configuration and putting into operation: "Specialist for call systems"

Procuring the required accessories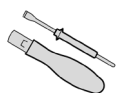

For the connection terminals:
Torque screwdriver
Blade: max. 2.6 x 0.6 mm
Tightening torque: max. 0.2 Nm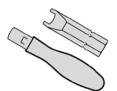

For front panel mounting:
Screwdriver with bit for
2-hole safety screws (size 4)


For programming the device:
FN 6700/00 Programming unit

Checking installation requirements

Before installation, ensure that the following requirements are met:

- The system bus is designed as IY(St)Y 4 x 2 x 0.8.
- The room bus is designed as IY(St)Y 2 x 2 x 0.6.
- All cables to be connected are de-energised.
- A flush-mounted switch box is installed: Ø 60 mm; depth min. 50 mm (a deeper switch box can make installation easier)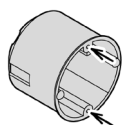


- For cavity wall boxes:
The fixing points for the front panel must be located at the top and bottom (not on the left and right).

Do not install the device until all installation requirements have been met.

Connecting, configuring and installing the device

Basic information on connecting to a bus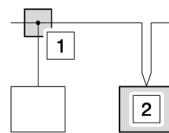

There are 2 options for connecting to a bus:

- 1 In a bus junction box
- 2 Looped through in the device to be connected


Basic information on handling the screw terminals

SAFETY INSTRUCTIONS

The small screw terminals can be damaged if force is applied.


Use a torque screwdriver with max. 0.2 Nm.

Connecting the device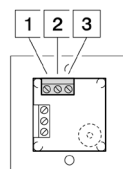

Connect the device to the system bus:


- 1 V_{PWR} (supply voltage)
- 2 GND (ground)
- 3 Data (system bus data)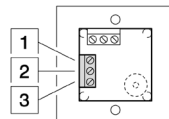

Connect the device to the room bus:


- 1 V_{PWR} (supply voltage)
- 2 GND (ground)
- 3 Data (room bus data)

Configuring the device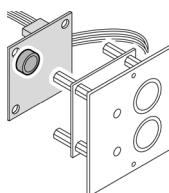

- 1 To access the programming connection for the configuration: Unscrew the piggyback board.
- 2 Have the system administrator carry out the following work:
 - Switch on the supply voltage. Control: The LEDs light up weakly (orientation light).
 - Configure the device at the programming connection.
 - Switch off the supply voltage again.
- 3 Fit the piggyback board.
- 4 Have the system administrator carry out the following work:
 - Switch on the supply voltage.
 - Integrate the device into the call system from the call centre.
- 6 Check all configurable functions.

Installing the device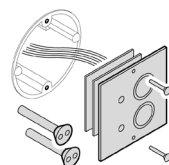

Mount the device on the switch box. Use the 2-hole safety screws included in the scope of delivery!
The device is now ready for use.

Instructions for operating staff

Keeping the device functional

Ensure that the work described in this section is carried out by a "specialist for call systems"

Conducting a quarterly inspection

Check for perfect condition and function:

- Call buttons and movable call triggering devices intended for use by residents or patients
- Signal lights and acoustic signalling devices

Conducting an annual inspection

Check for perfect condition and function:

- All connected devices for receiving, forwarding and displaying calls, e.g. plug-in devices for connecting call-triggering devices

Conducting annual maintenance

Carry out the following maintenance work:

- Maintain system components
- Install system updates
- Readjust and calibrate components and devices

Repair

If unauthorised deviations from the set status of the call system are detected:

- Carry out the repair within 24 hours.
- Record the repair in the operating log.

If the call system is completely or partially switched off for maintenance:

- Ensure that the affected rooms are otherwise checked until the call system or the switched-off part is switched on again.

Repairing the device

SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks.

There is a risk of malfunction if the device is repaired incorrectly.

Only have repairs carried out by the manufacturer.

About the functions of the device

This chapter describes the standard functions of the device. These functions can be changed in the central technology.


If your device behaves differently than described here, ask your system administrator to explain the changed functions.

Using the device

About the function of the buttons

These sensor buttons react to a light touch with the finger. They cannot be pressed down noticeably like conventional buttons.

Successful button use can be recognised by the reaction of the associated LED (orientation light/reassurance light).

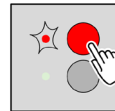Depending on the equipment of the device, the following functions are available:

Triggering call

Press the red call button.

The call is triggered.

The red reassurance light lights up.

Cancelling the call and setting staff attendance


Press the green attendance button.

The call is cancelled.

The red reassurance light turns off.

Staff attendance is set.

The green reassurance light lights up.

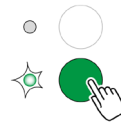Triggering an emergency call

With staff attendance set, also press the red call button.

The emergency call is triggered.

The red reassurance light flashes.

The green reassurance light lights up.

Recognising a forwarded call

(only for devices with buzzer)

As soon as you set the staff attendance, the green reassurance light lights up and the buzzer indicates a forwarded call:

- Every 10 seconds: normal call
- Every second: emergency call
- Every half second: alarm call

Deleting a forwarded call

(only for devices with buzzer)

Press the green attendance button to reset the staff attendance.

The green reassurance light turns off.

The buzzer falls silent.

Teaching residents or patients

Explain all the functions that are important to the resident or patient:

- Triggering a call

Instructions for auxiliary staff

Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

SAFETY INSTRUCTIONS

The device contains heat-sensitive components.
There is a risk of destruction when exposed to heat.
Do not sterilise the device thermally.

Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!