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
Original operating instructions


First steps

What do these operating instructions apply to?


These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features: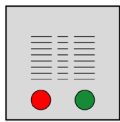

FN 6766/81
Voice terminal
Call/cancellation button
Aluminium design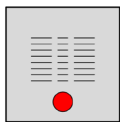

FN 6765/81
Voice terminal
Call button
Aluminium design

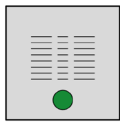FN 6762/81
Voice terminal
Cancellation button
Aluminium design


FN 6760/81
Voice terminal
Without buttons
Aluminium design

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate: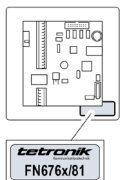


On the back of the front panel

Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of:

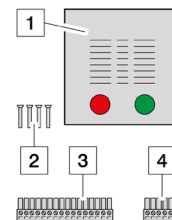- 1 Voice terminal
- 2 4 x 2-hole safety screw
- 3 Connector plug, 16-pin
- 4 Connector plug, 6-pin

Disposing of packaging

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
DANGER	Warns of imminent danger to life or danger of serious injury.
WARNING	Warns of possible imminent danger to life or danger of serious injury.
CAUTION	Warns of possible imminent danger of minor or slight injuries.
SAFETY INSTRUCTIONS	Warns of possible material damage.
NOTICE	Indicates additional information.

Information about the device


Intended use

The device is used in patients' and residents' rooms. Purpose depending on equipment:

- Triggering a call
- Triggering an emergency call
- Acknowledging a call and setting staff attendance
- Resetting staff attendance
- Establishing voice connections (speaking to individual beds)

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

Arrangement in the call system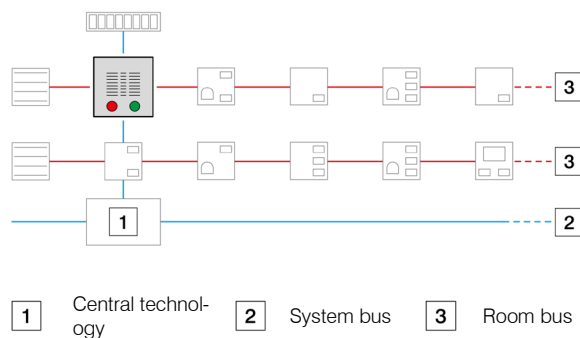

Features and properties

Similarities


- Including room electronics
- Speaker and microphone for voice communication
- Individual volume setting for each voice terminal (programmable via the control centre)
- Separate call priority for acoustic call triggering
- Voice communication with end devices of the TC system possible (fixed telephone, DECT telephone)
- Monitoring of all call units installed in the room
- Bed-by-bed call selection/speech
- Separate programmable call reset in the WC area in accordance with DIN VDE 0834
- Automatic feature for acoustic call triggering (configurable via call system management software)
- Contactless use of the green attendance/cancellation button via infrared transmitter
- Configuration via programming unit FN 6700/00
- Terminal strip for connection to the system bus including power supply
- Terminal strip for connection to the room bus (voice bus)
- Up to 8 call units can be connected to the beds
- Up to 4 different call modules per bed possible ($4 \times 8 = 32$ modules)
- Connection option for up to 8 call units in the WC area

Type-specific features


- FN 6766/81: Red call button and green cancellation button
- FN 6765/81: Red call button
- FN 6762/81: Green cancellation button
- FN 6760/81: Without buttons

Controls and displays


FN 6766/81 Voice terminal with call/cancellation button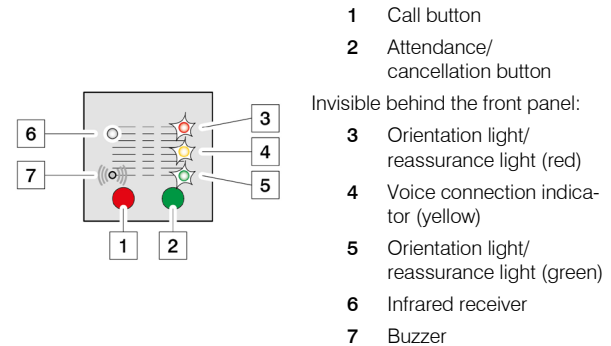


FN 6765/81 Voice terminal with call button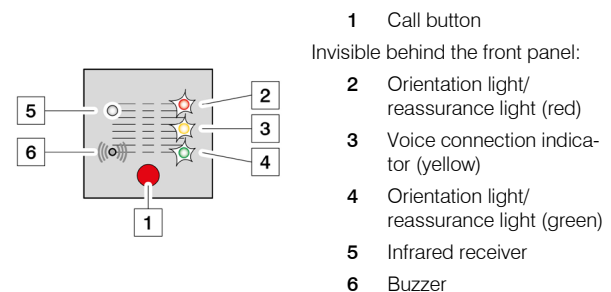


FN 6762/81 Voice terminal with cancellation button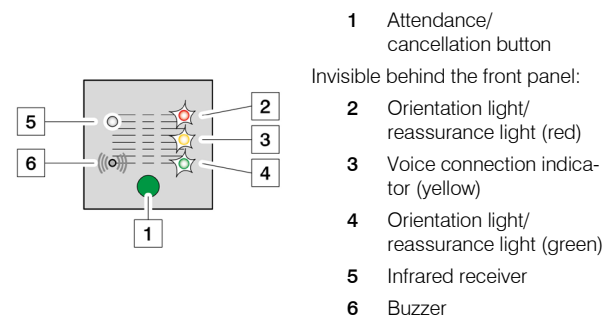

FN 6760/81 Voice terminal without buttons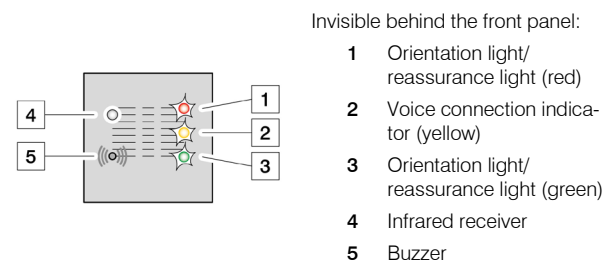

For all voice terminals: tamper detection

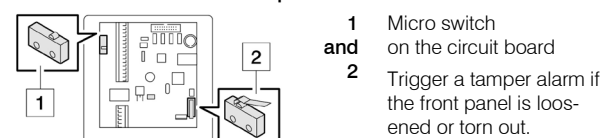Instructions for technical staff

Technical data

Physical data

Dimensions (W x H x D)	140 x 140 x 55 mm
Weight	
without housing	370 g
with housing	550 g
Front panel material	Aluminium 6082

Electrical data

Supply voltage	18...28 V DC
Input current system bus	30...700 mA
Protection class	IP 20

Ambient conditions

Permissible ambient temperature	+10...+40°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device must not be used in potentially explosive atmospheres.

The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information


- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

Ensuring staff qualification


Ensure that all work is carried out by suitable staff:

- Installation and connection:
Qualified electrician or trained assistant
- Putting into operation: "Specialist for call systems"


Procuring the required accessories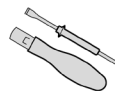

For the connection terminals:
Torque screwdriver
Blade: max. 2.6 x 0.6 mm
Tightening torque: max. 0.2 Nm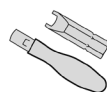

For front panel mounting:
Screwdriver with bit for
2-hole safety screws (size 4)

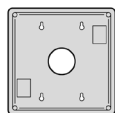FN 676x/81
Flush-mounted housing for voice terminal,
aluminium design, colour white

For programming the device:
FN 6700/00 Programming unit

Procuring recommended accessories

For contactless use of the green attendance/cancellation button:
FN 6700/11
Infrared transmitter for room bus modules

Checking installation requirements


Before installation, ensure that the following requirements are met:

- The system bus is designed as IY(St)Y 6 x 2 x 0.8.
- The room bus is designed as IY(St)Y 2 x 2 x 0.6.
- All cables to be connected are de-energised.

Do not install the device until all installation requirements have been met.

Mounting the flush-mounted housing

- ⚠ DANGER** Concealed cables or pipes possible.
 Danger to life if power lines or gas pipes are damaged.
 Search the installation site with a cable finder before chiselling.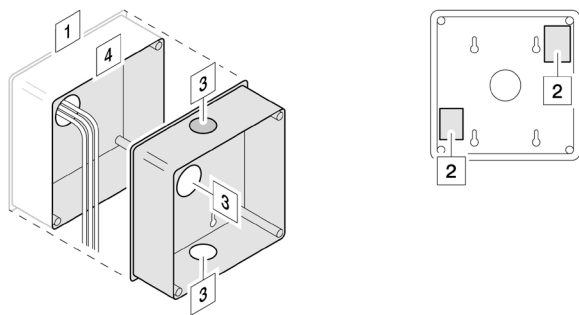

- 1 Create an opening for the flush-mounted housing:
145 x 145 x 55 mm plus allowance for plastering
- 2 Ensure that the flush-mounted housing is correctly aligned:
The abutments for the micro switches must be positioned at the bottom left and top right.
- 3 Open the required cable aperture and feed the connection cables through.
- 4 Fix the flush-mounted housing flush with the wall by plastering.

Configuring, connecting, programming and installing the device

Configuring the device

Configure the contact type of the inputs:

- | | |
|-------|-------------------------|
| 1 OFF | Vital sign call input |
| 2 ON | Normally closed contact |

- | | |
|-------|-----------------------|
| 1 ON | Vital sign call input |
| 2 OFF | Normally open contact |

Door contact switching inputs:

- | | |
|-------|-------------------------|
| 3 OFF | Door 1: |
| 4 ON | Normally closed contact |


- | | |
|-------|-----------------------|
| 3 ON | Door 1: |
| 4 OFF | Normally open contact |

- | | |
|-------|-------------------------|
| 5 OFF | Door 2: |
| 6 ON | Normally closed contact |

- | | |
|-------|-----------------------|
| 5 ON | Door 2: |
| 6 OFF | Normally open contact |

- | | |
|-------|-------------------------|
| 7 OFF | Door 3: |
| 8 ON | Normally closed contact |

- | | |
|-------|-----------------------|
| 7 ON | Door 3: |
| 8 OFF | Normally open contact |

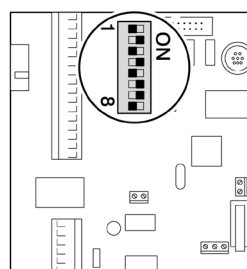Note: To keep the voice terminal visually unobtrusive, individual LEDs can be blocked.

Configure the function of the LEDs:

- | | |
|------|----------------|
| 1 ON | Red LED active |
| 2 ON | Red LED active |


- | | |
|-------|-----------------|
| 1 OFF | Red LED blocked |
| 2 OFF | Red LED blocked |

- | | |
|------|------------------|
| 3 ON | Green LED active |
| 4 ON | Green LED active |

- | | |
|-------|-------------------|
| 3 OFF | Green LED blocked |
| 4 OFF | Green LED blocked |


- | | |
|------|---------------------------------|
| 5 ON | Yellow LED active ^{*)} |
|------|---------------------------------|

- | | |
|--------|------------------|
| 6 to 8 | without function |
|--------|------------------|

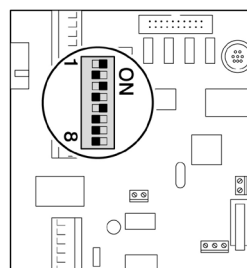^{*)} The yellow LED must not be blocked.

Basic information on connecting to a bus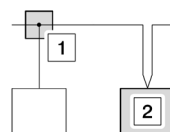

There are 2 options for connecting to a bus:

- 1 In a bus junction box
- 2 Looped through in the device to be connected

Basic information on handling the screw terminals


SAFETY INSTRUCTIONS

The small screw terminals can be damaged if force is applied.

Use a torque screwdriver with max. 0.2 Nm.


Connecting the device

Connect the device to the system bus using the plug connector: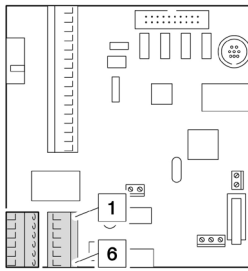

- 1 V_{PWR} (supply voltage)
- 2 GND (ground)
- 3 Data 1
- 4 Data 2
- 5 Audio 1
- 6 Audio 2


Connect the device to the room bus using the plug connector :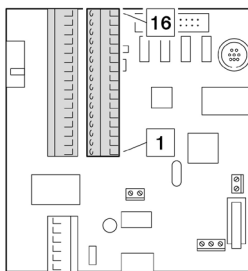

- 1 V_{PWR} (supply voltage)
- 2 GND (ground)
- 3 Data 1
- 4 Data 2
- 5 Audio 1
- 6 Audio 2
- 7 V_{PWR} (supply voltage)
- 8 GND (ground)
- 9 Vital sign call input^{*)}
- 10 V_{PWR} (supply voltage)
- 11 Door cont. switching input 1^{*)}
- 12 V_{PWR} (supply voltage)
- 13 Door cont. switching input 2^{*)}
- 14 Door cont. switching input 3^{*)}
- 15 Signal input A^{*)}
- 16 Signal input B^{*)}


^{*)} Observe the specifications for the inputs as follows:

Specifications for the vital sign call input: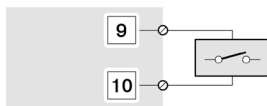


+18.0...24.0 V
0.5 mA (at 24.0 V)
For the contact type, see
"Configuring the device", page 4.

Specifications for the door contact switching inputs: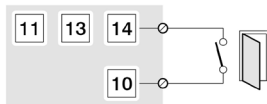


+18.0...24.0 V
0.5 mA (at 24.0 V)
For the contact type, see
"Configuring the device", page 4.

Specifications for the signal input (e.g. fire detector):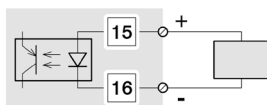


Optocoupler input
+10.0...24.0 V
4.6 mA (at 24.0 V)

Programming the device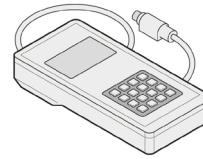


The device is then programmed using the programming unit FN 6700/00.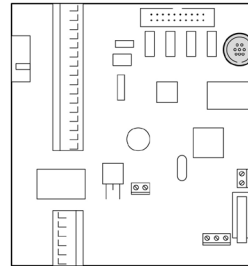

- 1 Switch on the supply voltage. The device is then ready for programming.
Feature: The LEDs light up weakly (orientation light).
- 2 Connect the programming unit to the device.

- 3 Programme the device in accordance with the operating instructions for the programming unit.
- 4 After programming is complete: Remove the programming unit.

Installing the device

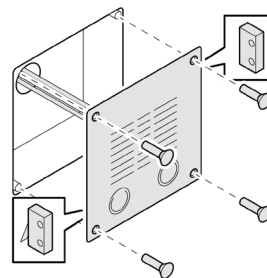Pay attention to the cable routing when attaching the front panel: The cables must not interfere with the function of the micro switches!
Fasten the front panel with the 4 safety screws supplied.

Checking commissioning requirements

Before commissioning, ensure that the following requirements are met:

- The device is connected to the call system.
- The call system is active.

Only put the device into operation if all commissioning requirements have been met.

Putting the device into operation

Notify the system administrator so that they can connect the device to the call system from the call centre.

Instructions for operating staff

Keeping the device functional

Ensure that the work described in this section is carried out by a "specialist for call systems"

Conducting a quarterly inspection

Check for perfect condition and function:

- Call buttons and movable call triggering devices intended for use by residents or patients
- Signal lights and acoustic signalling devices

Conducting an annual inspection

Check for perfect condition and function:

- All other devices for call triggering, call cancellation and attendance reporting
- All other display devices

Conducting annual maintenance

Carry out the following maintenance work:

- Maintain system components
- Install system updates

Repair

If unauthorised deviations from the set status of the call system are detected:

- Carry out the repair within 24 hours.
- Record the repair in the operating log.

If the call system is completely or partially switched off for maintenance:

- Ensure that the affected rooms are otherwise checked until the call system or the switched-off part is switched on again.

Repairing the device

SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks.

There is a risk of malfunction if the device is repaired incorrectly.

Only have repairs carried out by the manufacturer.

About the functions of the device

This chapter describes the standard functions of the device. These functions can be changed in the central technology. Example: The functions of the LEDs may be blocked.

Ask the system administrator how the device is configured.

Using the device

The use of the individual functions depends on the equipment of the device.

Triggering a call

Press the red call button.

The call is triggered.

The red reassurance light lights up.

Cancelling the call and setting staff attendance

Press the green attendance button.

The call is cancelled.

The red reassurance light turns off.

Staff attendance is set.

The green reassurance light illuminates.

Triggering an emergency call

With staff attendance set, also press the red call button.

The emergency call is triggered.

The red reassurance light flashes.

The green reassurance light lights up.

Recognising a forwarded call

As soon as you set the staff attendance: The green reassurance light lights up and the buzzer indicates a forwarded call:

- Every 10 seconds: normal call
- Every second: emergency call
- Every half second: alarm call

Deleting a forwarded call

Press the green attendance button.

The green reassurance light turns off.

The buzzer falls silent.

The staff attendance is reset.

Recognising a voice connection

When an acoustic signal sounds and the yellow LED lights up:

A voice connection has been established.

Instructions for auxiliary staff

Using the infrared transmitter

For contactless use of the green attendance/cancellation button:
Use the infrared transmitter.

Using acoustic call triggering

Acoustic call activation is used to monitor a room.

Application example: A room is not to be entered during certain times. In the event of non-compliance, the acoustic events that occur should trigger a call.

Have this function enabled by the system administrator. The parameters can be configured in the call system management software:

- Activation schedule
- Volume response threshold
- Response delay

Teaching residents or patients

Explain all the functions that are important to the resident or patient:

- Sending a call

Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

SAFETY INSTRUCTIONS

The device contains heat-sensitive components.
There is a risk of destruction when exposed to heat.

Do not sterilise the device thermally.

Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!