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Original operating instructions

### First steps

#### What do these operating instructions apply to?

These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features:

FN 6864/21 Motion detector for room radio

FN 6064/21 Motion detector for room bus

#### Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate:


On the underside of the device

#### Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of: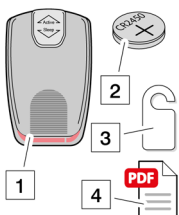

- 1 Motion detector
- 2 Battery CR2450 (installed ready for use)
- 3 Door hanger
- 4 Download the latest version of the operating instructions at <https://www.tetronik-kommunikationstechnik.de/downloads/original-betriebsanleitungen/>

Search term: FN number of your device.

#### What to do with the packaging?

If the device or parts of it are to be stored before or after use: Keep the packaging.

If nothing is to be stored:

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

#### Using the operating instructions

##### Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

##### Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
<b>DANGER</b>	Warns of imminent danger to life or danger of serious injury.
<b>WARNING</b>	Warns of possible imminent danger to life or danger of serious injury.
<b>CAUTION</b>	Warns of possible imminent danger of minor or slight injuries.
<b>SAFETY INSTRUCTIONS</b>	Warns of possible material damage.
<b>NOTICE</b>	Indicates additional information.

## Information about the device


### Intended use

The motion detector is used to protect those who are disoriented. It detects movements in the detection area and triggers a call in the call system via an associated receiver.

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

**⚠ DANGER** The use of this device does not release the staff from their duty to supervise the resident or patient.  
 This device is not intended for reporting life-threatening conditions.

### Arrangement in the call system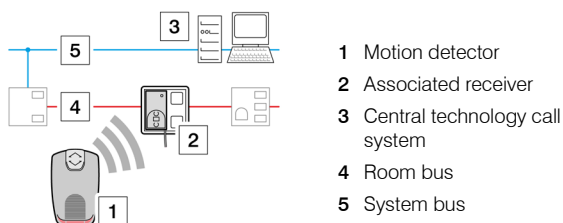


### Features and properties

- Flat housing design – not a trip hazard
- Non-slip housing
- Simple activation/deactivation via foot-operated slide switch

### Controls and displays

The device has the following controls and displays:

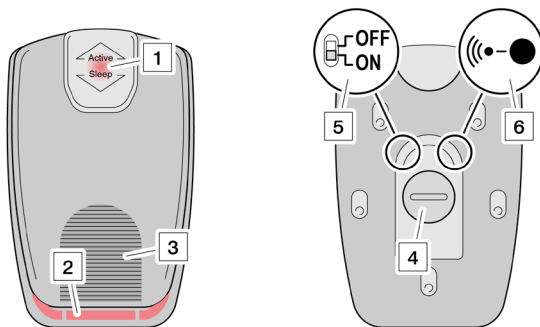#### On the top:

- 1 Call indicator (red LED)
- 2 Indicator area for operating status
- 3 Foot slide switch

#### On the underside:

- 4 Battery compartment cover
- 5 On/off switch
- 6 Registration button

### Technical data

#### Physical data

Dimensions (W x H x D)	120 x 12 x 180 mm
Weight	310 g
Step load capacity of the foot-operated slide switch	180 kg

#### Electrical data

Power supply	Battery CR2450 3 V 560 mAh
Battery life	Approx. 2 years
Operating frequency	433.92 MHz
Protection class	IP 53
Detection area of the motion sensor	Range: 5 m Horizontal angle: 60° Vertical angle: 80°
Range of the transmitter	Up to approx. 10 m inside buildings

#### Ambient conditions

Permissible ambient temperature	0... +40°C
Permissible storage temperature	-10... +60°C
Relative humidity	10...85% (without condensation)

The device is suitable for use in damp conditions to some extent, e.g. in wet rooms.

The device is only suitable for use in shielded rooms (e.g. X-ray rooms) if the associated receiver is located in the same room.

The device must not be used in potentially explosive atmospheres.


The declaration of conformity can be requested from the manufacturer. See last page for contact details.

### Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

## Instructions for technical staff

### Procuring the required accessories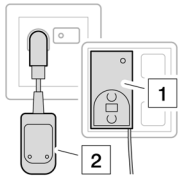


- 1 FN 6864/00  
IMA receiver for room radio
- 2 Receiver for wireless handheld button  
FN 6064/10 7-pole DIN  
FN 6064/50 8-pole mini-DIN

### Putting the device into operation

Ensure that the following requirements are met:


- The operating instructions for the corresponding receiver are to hand.
- The associated receiver operates on the same frequency as the motion detector.
- The associated receiver is ready for use and is within range of the motion detector.

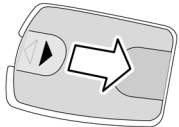- 1 Move the foot slide switch backwards towards "Sleep". This makes the controls on the underside accessible.

- 2 Switch on the device.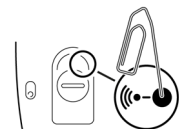

- 3 Press the registration button.  
The device sends a call and the call indicator (red LED) flashes 5 times.
- 4 Continue the registration process as described in the operating instructions for the receiver.  
The motion detector is then ready for use.

### Replacing the battery


Battery type: CR2450

The battery compartment is located on the underside of the device.



- 1 Open the battery compartment cover with a coin.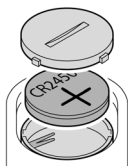

- 2 Insert the new battery.  
Ensure correct polarity (positive pole at the top).

- 3 Close the battery compartment cover.

### De-registering the device

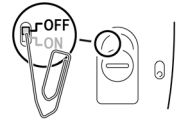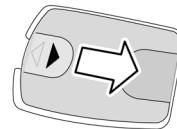If the device is no longer needed on the associated receiver:

- 1 Move the foot slide switch backwards towards "Sleep". This makes the controls on the underside accessible.
- 2 Switch off the device.
- 3 Continue the de-registration process as described in the operating instructions for the receiver.

The motion detector is then de-registered.

### Servicing and maintaining the device

Ensure that this device and the entire call system are fail-safe at all times. Follow the regulation DIN VDE 0834-1:2016-06 (11.2 to 11.4).

### Repairing the device

#### SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks.

There is a risk of malfunction if the device is repaired incorrectly.

Only have repairs carried out by the manufacturer.


## Instructions for operating staff

### **⚠ DANGER**


The use of this device does not release the staff from their duty to supervise the resident or patient.

This device is not intended for reporting life-threatening conditions.

### Setting up and using the device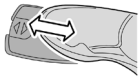

- 1 Place the device on the floor and check that it is stable. The device must not move when the foot-operated slide switch is operated.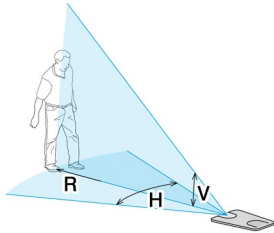


- 2 Align the device with the desired detection area:

R Range: max. 5 metres

H Horizontal angle: 60°

V Vertical angle: 80°

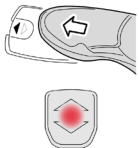- 3 To activate the device: Push the foot slide switch forwards towards "Active". The device triggers a call in the call system whenever movement is detected. Call display: The red LED flashes 5 times.

- 4 Test the function and correct the alignment if necessary.

When staff enter the room: Deactivate the device to avoid triggering a call.

- 5 Move the foot slide switch backwards towards "Sleep".

The device does not detect any movements and does not trigger any calls.

When leaving the room:

- 6 Activate the device again.

## Instructions for auxiliary staff

### Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

In case of heavy soiling: Clean the device with a Hoover and an upholstery brush.

### Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

#### SAFETY INSTRUCTIONS

The device contains heat-sensitive components.

There is a risk of destruction when exposed to heat.

Do not sterilise the device thermally.

### Procure consumables

#### SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks.

There is a risk of malfunction if foreign material is used.

Only use original material from the manufacturer.

The following consumables are available:

Battery type: CR2450

### Storing the device when not in use

Store the device in its original packaging in a suitable storage location. For the requirements for the storage location, see "Technical data".

### Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!