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Original operating instructions

| First steps

What do these operating instructions apply to?

These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features:

- FN 6864/22 Call pad for room radio
- FN 6064/22 Call pad for room bus

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate:


On the electronics unit
(visible when the zip is open)

Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of: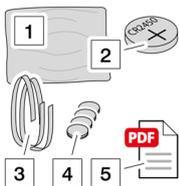

- 1 Call pad
- 2 Battery CR2450
(installed ready for use)
- 3 2 x Velcro tape
- 4 4 x Velcro adhesive pad
- 5 Download the latest version of the operating instructions at <https://www.tetronik-kommunikationstechnik.de/downloads/original-betriebsanleitungen/>. Search term: FN number of your device.

What to do with the packaging?

If the device or parts of it are to be stored before or after use: Keep the packaging.

If nothing is to be stored:

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
DANGER	Warns of imminent danger to life or danger of serious injury.
WARNING	Warns of possible imminent danger to life or danger of serious injury.
CAUTION	Warns of possible imminent danger of minor or slight injuries.
SAFETY INSTRUCTIONS	Warns of possible material damage.
NOTICE	Indicates additional information.


Information about the device

Intended use

The call pad enables residents or patients with severe motor impairments to trigger a call by pressing lightly on the call pad.

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

Arrangement in the call system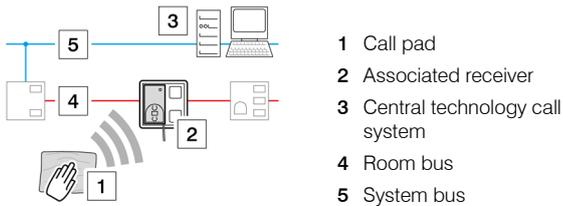


Features and properties

- Pneumatic call triggering with a light touch
- Wide range of fastening options using Velcro tape
- Acoustic call display

Controls and displays

The device has the following controls and displays: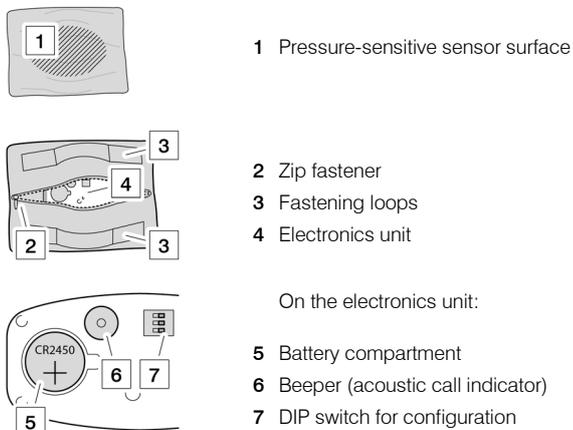

Technical data

Physical data

Pad dimensions (W x H x D)	225 x 150 x 45 mm
Pressure area dimensions (W x H)	180 x 100 mm
Weight	170 g
Colour	Orange

Electrical data

Supply voltage	Battery CR2450 3 V 570 mAh
Battery life	Approx. 3 years
Operating frequency	433.92 MHz
Protection class	IP 30
Transmitter range	approx. 10 metres

Ambient conditions

Permissible ambient temperature	0...+40°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device is only suitable for use in shielded rooms (e.g. X-ray rooms) if the associated receiver is located in the same room.

The device must not be used in potentially explosive atmospheres.


 The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

Instructions for technical staff

Procuring the required accessories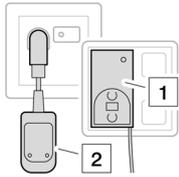


- 1 FN 6864/00
IMA receiver for room radio
- 2 Receiver for wireless handheld button
FN 6064/10 7-pole DIN
FN 6064/50 8-pole mini-DIN

Putting the device into operation

Ensure that the following requirements are met:

- The operating instructions for the corresponding receiver are to hand.
- The associated receiver operates on the same frequency as the call pad.
- The associated receiver is ready for use and is within reach of the call pad.

Configuring the device

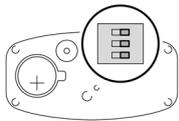
The device is configured using the DIP switches on the electronics unit. Make the settings as follows:


- 1 Switch on the device.

- 2 Should the device beep 5 times when a call is triggered?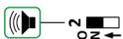


ON Switch on the beeper.

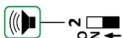OFF Switch off the beeper.

- 3 Should the call pad trigger a call when touched?

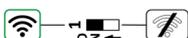ON Switch on the call triggering function.

OFF Switch off the call triggering function.

Registering the device on the receiver


- 1 Trigger a call.
The device beeps 5 times (call display).
- 2 Continue the registration process as described in the operating instructions for the corresponding receiver.
The call pad is then ready for use.


Replacing the battery

Battery type: CR2450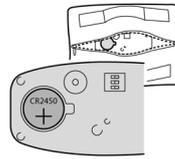

The battery is located in the battery compartment of the electronics unit.

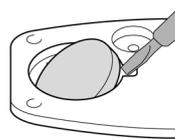
- 1 Lever out the battery.

- 2 Insert the new battery.
Ensure correct polarity (positive pole at the top).

De-registering the device

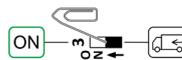If the device is no longer needed on the associated receiver:

- 1 Switch off the device.
- 2 Continue the de-registration process as described in the operating instructions for the corresponding receiver.
The call pad is then de-registered.

Repairing the device

The device fulfils safety-relevant tasks.

There is a risk of malfunction if the device is repaired incorrectly.

Only have repairs carried out by the manufacturer.


Instructions for operating staff

Attaching the device

- 1 Find a suitable position for the call pad together with the resident or patient.

The resident or patient should be able to reach the call pad easily according to their motor skills, e.g. with a hand, elbow or knee movement.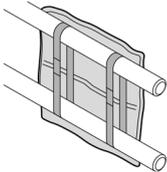

- 2 Attach the call pad so that it remains securely in position when activated.

Use the Velcro strips or Velcro adhesive pads included in the scope of delivery to attach it.

- 3 Have the resident or patient test the function.
Correct the position if necessary.

Triggering a call

- Touch the call pad.
The call is triggered.
Call display: The beeper sounds 5 times.

Teaching residents or patients

Explain all the functions that are important to the resident or patient:

- Sending a call

Instructions for auxiliary staff

Procuring consumables

SAFETY INSTRUCTIONS

The device fulfils safety-relevant tasks.

There is a risk of malfunction if foreign material is used.

Only use original material from the manufacturer.

The following consumables are available:


- ET013432 Spare set, consisting of:
- 1 x cushion cover
 - 1 x inner fleece
 - 2 x Velcro tape
 - 4 x Velcro adhesive pad


Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

In the event of heavy soiling, we recommend removing the electronics unit before intensive cleaning or replacing the cushion cover.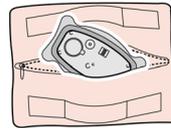

- 1 Remove the electronics unit from the call pad.

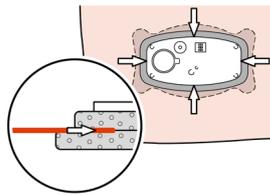- 2 When inserting the electronics unit, ensure that the fit is airtight:

Insert the orange fabric all around between the grey foam layers.

Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

SAFETY INSTRUCTIONS

The device contains heat-sensitive components.

There is a risk of destruction when exposed to heat.

Do not sterilise the device thermally.

Storing the device when not in use

Store the device in its original packaging in a suitable storage location. For the requirements for the storage location, see "Technical data".

Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!