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
FN 6000

In a nutshell

Operation and functions at a glance

Interpreting LED indicators

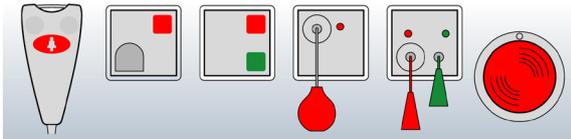Devices with an LED indicator


Orientation light in the button

In idle state:
The LED lights up dimly.

Reassurance light in the button


After pressing the button:
The LED lights up brightly.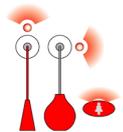

Other LED indicators

Some devices have illuminated controls or separate LEDs with identical functions.

Triggering emergency/ doctor call

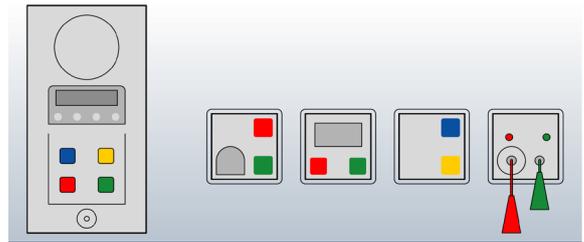Devices with the option of triggering an emergency call and a doctor call

Triggering an emergency call


If staff attendance is set:
The green LED lights up brightly.
Press the red call button.
The red LED flashes.
The emergency call is triggered.

Triggering a doctor call

If staff attendance is set:
The green LED lights up brightly.
Press the blue doctor call button.
The blue LED flashes.
The doctor call is triggered.

Triggering a call

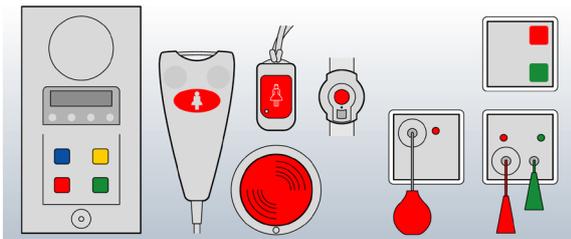
Devices with the option of call triggering

With the call button

Press the red call button.
The red LED in the call button lights up brightly.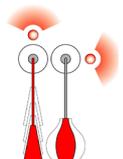

With the pull cord switch or pneumatic bellows


Pull the red pull cord switch or press the red pneumatic bellows together.
The corresponding red LED lights up brightly.

With other call triggers

Press the red control.
The red control or the corresponding red LED flashes up to 5 times.

Setting staff attendance and cancelling a call/emergency call/doctor call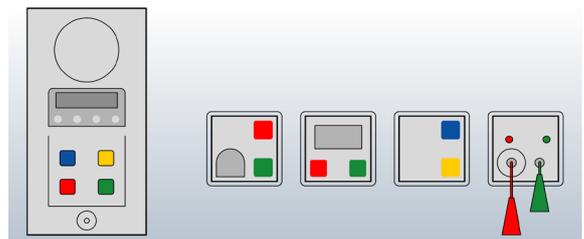

Devices with the option of setting staff attendance and cancelling call

Setting staff attendance


Press the green button.
The green LED lights up brightly.
Staff attendance is set.
The call/emergency call is cancelled.

Setting 2nd staff attendance

Press the yellow button.
The yellow LED lights up brightly.
The 2nd staff attendance is set.
The doctor call is cancelled.

Recognising and cancelling the tear-off alarm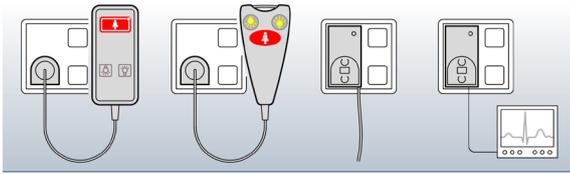

Devices that can trigger a tear-off alarm

A tear-off alarm is triggered automatically as soon as an operating device is unplugged for longer than 10 seconds (e.g. IMA handheld button, IMA receiver or IMA diagnostic adapter).


Recognising a tear-off alarm: The red LED flashes.


To cancel the tear-off alarm: Reconnect the same operating device.

To de-register the operating device: Press the green cancellation button for 3 seconds. The red LED turns off.

Interpreting the call indicator outside the room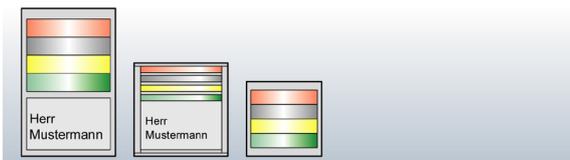


Call indicators outside the room

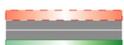-  **Illuminated red**
A call is pending.
-  **Slowly flashing red and illuminated yellow**
An emergency call is pending.
-  **Slowly flashing red and illuminated green**
An emergency call is pending.
-  **Illuminated red and white**
A wet room call is pending.
-  **Illuminated yellow**
Staff have set the 2nd staff attendance.
-  **Illuminated green**
Staff have set the staff attendance.
-  **Illuminated yellow and green**
Staff have set the staff attendance and the 2nd staff attendance.
-  **Fast flashing red**
An alarm call is pending.

Origin of an alarm call: Connected device (e.g. medical device) or connected system (e.g. fire alarm system).

Recognising a forwarded call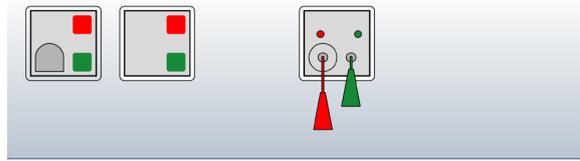

Devices that can acoustically signal a forwarded call

Practical example: While staff attendance is set in one room, another call is triggered in another room on the same ward. This call is forwarded to the room where staff are present and signalled acoustically.



With staff attendance set:

The green LED lights up and the buzzer signals the forwarded call.


Recognising origin and call type

Note: The acoustic signalling depends on the programme and may differ from the examples shown. Ask the system administrator.

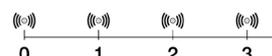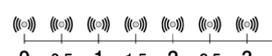-  1 signal every 10 seconds:
Room or wet room call
-  1 signal per second:
Emergency call
-  1 signal every 0.5 seconds:
Alarm call


Origin of an alarm call: Connected device (e.g. medical device) or connected system (e.g. fire alarm system).

Reading alphanumeric indicators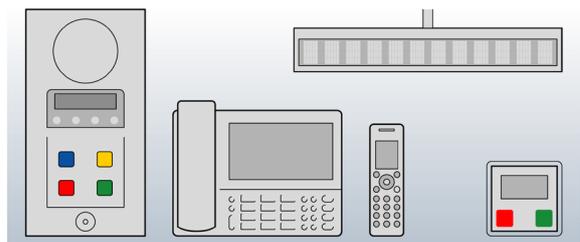

Devices with alphanumeric call indicator

Alphanumeric indicators show the call type and call origin. Depending on the device and programming, further call details can also be displayed.

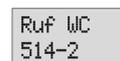 Example: Staff room display or display of the room electronics module


 Example: Corridor display with 2-line, continuously changing display

 Example: Display of the IP ward console

Talking to a room

On the IP ward console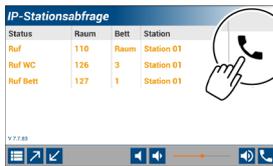

For incoming calls:
Tap the handset.

The voice connection is established and is signalled acoustically in the room.

→ When the red "Speak" button is displayed:
Continue with "Listening and speaking with the speak button".

→ When the green "Listen" button is displayed:
Continue with "Listening and speaking with the shift key".

Listening and speaking with the speak button

As long as the button is red, you will hear the room.

To speak to the room:
Press and hold the red "Speak" button.

The pressed button turns green.
Press and hold the button to speak to the room.

To listen to the room:
Release the button.

The button turns red again.

Continue with "Cancelling a call and disconnecting a voice connection".

Listening and speaking with the shift key

To listen to the room:
Briefly tap the green "Listen" button.

The button turns purple.
You can hear the room.


To speak to the room:
Briefly tap the purple "Speak" button.

The button turns green again.
Speak to the room.

Continue with "Cancelling a call and disconnecting a voice connection".

Cancelling a call and disconnecting a voice connection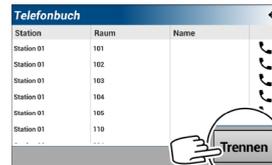

To disconnect the voice connection: Tap "Disconnect".

Note: Disconnecting does not cancel the call^{*)}.


To cancel the call:
Tap "Cancel" before disconnecting^{*)}.

(Note: The room number is displayed in the bottom left).

^{*)} The option to cancel a call remotely depends on the programme.
Ask the system administrator.

On the staff room display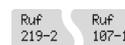

The display shows all pending calls one after the other in an endless loop.

Calls with the option of voice communication are marked with the loudspeaker symbol.

Use the arrow buttons to select a call with the loudspeaker symbol.

To establish a voice connection:
Press the loudspeaker button.

The loudspeaker button flashes. The voice connection is signalled acoustically in the called room. You can hear the room.

To speak to the room:
Press the loudspeaker button.

To listen to the room:
Release the loudspeaker button.

To disconnect the voice connection:
Briefly press the reset button.

To cancel the call:
Press the reset button for at least 5 seconds.