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Original operating instructions

First steps

What do these operating instructions apply to?

These operating instructions apply to the following devices, which differ in terms of their range of functions or equipment features:

	FN 6812/00	Cancellation button
	FN 6816/00	Call button and cancellation button
	FN 6816/60	Call/cancellation button, NFC
	FN 6813/00	Call button, cancellation button and button for 2nd attendance
	FN6810/01	Call button, cancellation button and vital sign call button
	FN 6810/00	Call button and cancellation button with IMA
	FN 6819/00	Call button and cancellation button with display

Identifying the device

Make sure that these operating instructions belong to your device. These operating instructions do not apply to other devices, even if they look similar or appear to be of the same design.

Unique feature: Type plate ("x" = any number)

Position of the type plate:

On the front of the installation frame

Checking the scope of delivery

Check that the scope of delivery is complete and in perfect condition. In case of missing parts or damage: Do not connect the device and do not put it into operation! Make a complaint to the supplier.

The scope of delivery consists of:

	Electronics with control panel
	Installation frame
	Connector plug for the system bus

Disposing of packaging

Dispose of the packaging in accordance with regional regulations. Do not dispose of the packaging in household waste!

Using the operating instructions

Read first!

Read these operating instructions in full. Keep the operating instructions in a safe place. If you have any questions, please contact the manufacturer.

Interpreting signal words

Pay attention to signal words that warn of dangers. Follow the instructions for risk avoidance.

Signal word	Meaning
 DANGER	Warns of imminent danger to life or danger of serious injury.
 WARNING	Warns of possible imminent danger to life or danger of serious injury.
 CAUTION	Warns of possible imminent danger of minor or slight injuries.
 SAFETY INSTRUCTIONS	Warns of possible material damage.
 NOTICE	Indicates additional information.

Information about the device

Intended use

The device is used in a resident's or patient's room. Purpose (depending on equipment):


- Triggering calls
- Setting 2nd attendance
- Cancelling calls
- Reading information on pending calls

For devices with IMA (intelligent solenoid connection) additionally:

- Connecting a patient operating device, e.g. handheld button or multiple handheld button
- Connecting another signalling device, e.g. receiver for room radio or diagnostic adapter

Any other use is contrary to the intended use and may lead to malfunctions and damage to this and connected devices. Disregarding these operating instructions also constitutes improper use.

Arrangement in the call system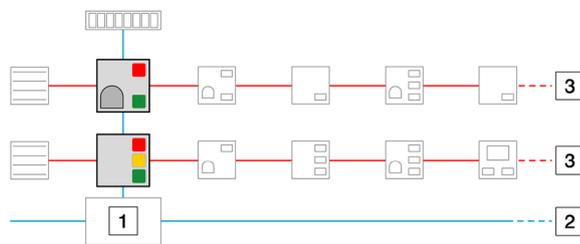

- 1 Central technology 2 System bus 3 Room bus

Features and properties

Similarities

- Flush-mounted design for installation in a switch box
- Suitable for switch ranges with 55 mm internal dimensions, e.g.:
 - GIRA Standard 55
 - GIRA E2
 - GIRA Event
 - Other suitable switch ranges on request
- Antimicrobial surface, easy to clean
- Configurable performance features
 - via call system management software from version 7.x
 - via remote configuration
- Firmware update possible during ongoing operation via call system management software from version 7.x
- Data transmission and power supply via the FN 6000® system bus
- Connection to the room bus II to the call units via 3-pole terminal strip
- Buzzer for acoustic signalling of forwarded calls

FN 6810/00: Call button and cancellation button with IMA

- Connection option for patient operating devices or other signalling devices via IMA (intelligent solenoid connection)
 - Fast, secure connection
 - Non-destructive disconnection in the event of strong cable tension
 - Tear-off report to the central technology

FN 6819/00: Call button and cancellation button with display

- Display of the type and origin of pending calls

FN 6816/60: Call/cancellation button, NFC

- Contactless use of the green attendance/cancellation button via NFC transponder

Controls, operator connections and displays

Depending on the equipment:

-  1 Reassurance light
-  2 Call button with orientation light (backlighting)
-  3 Reassurance light
-  4 2nd attendance/cancellation button with orientation light (backlighting)
-  5 Reassurance light
-  6 Vital sign call button
-  7 Reassurance light
-  8 Attendance/cancellation button with orientation light (backlighting)
-  9 NFC sensor surface
-  10 IMA socket for patient operating device or other signalling device
-  11 Display for the indication of calls

Technical data

Physical data

Dimensions (W x H x D)	
Control panel complete with frame	56 x 56 x 8 mm 80 x 80 x 10 mm
Colour	Pure white (similar to RAL 9010)

Electrical data

Supply voltage	18...28 V DC
Input current	min. 9 mA; max. 59 mA
Output current signal lamps and impulse switches (open collector outputs)	max. 100 mA
Protection class	IP 20

Ambient conditions

Permissible ambient temperature	0...+40°C
Permissible storage temperature	-10...+60°C
Relative humidity	10...85% (without condensation)

The device is only suitable for use in dry indoor areas. It must not be exposed to moisture.

The device must not be used in potentially explosive atmospheres.

 The declaration of conformity can be requested from the manufacturer. See last page for contact details.

Normative information

- Planning and testing of the call system and certification of its functionality must be carried out by a "specialist planner for call systems".
- This device is not a medical device within the meaning of Directive 93/42/EEC.
- The electrical safety of the call system is guaranteed by system separation.

Instructions for technical staff

Ensuring staff qualification

Ensure that all work is carried out by suitable staff:


- Installation and connection:
 - Qualified electrician or trained assistant
- Putting into operation: "Specialist for call systems"

Procuring the required accessories


FN 6730/01 Cover frame

Suitable alternatives:
 GIRA Standard 55
 GIRA E2
 GIRA Event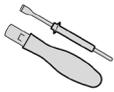

For the connection terminals:
 Torque screwdriver
 Blade: max. 2.6 x 0.6 mm
 Tightening torque: max. 0.2 Nm

For devices to which a multiple handheld button is to be connected, additionally:

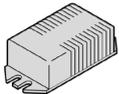Impulse switch
 in accordance with DIN EN 60601

Procuring recommended accessories

Remote control for contactless use of the green attendance/cancellation button:
 FN 6801/10 NFC transponder

Checking installation requirements

Before installation, ensure that the following requirements are met:

- The system bus is designed as IY(St)Y 4 x 2 x 0.8.
- The room bus is designed as IY(St)Y 2 x 2 x 0.6.
- All cables to be connected are de-energised.

Do not install the device until all installation requirements have been met.

Documenting the serial number


To simplify documentation, each device has a two-part sticker with the serial number.

- Obtain the serial number list from the system administrator.
- Detach the flag with the serial number.
Leave the part with the QR code on the device.
- Stick the flag with the serial number in the corresponding column. Examples:
 ZE1 = Room electronics 1
 Zisi = Room signal light
 BT1 = Bed 1
 WC1 = Toilet 1
 WC2 = Toilet 2
 ZI = Room
 ZP = Additional circuit board
- For multiple identical devices per room:
Set up additional columns and number the columns consecutively. Example: "ZI 1", "ZI 2" etc.

Installing and connecting the device

Basic information on connecting to a bus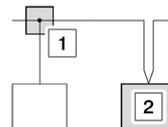

There are 2 options for connecting to a bus:

- In a bus junction box
- Looped through in the device to be connected

Basic information on handling the screw terminals


Small screw terminals.

Risk of damage when force is applied.


Use a torque screwdriver.

Tightening torque: max. 0.2 Nm


Installing and connecting devices without a display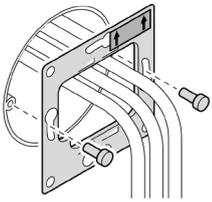

- 1 Feed the cables through the installation frame.
 - 2 Fit the installation frame.
- Important: Type plate sticker top right!

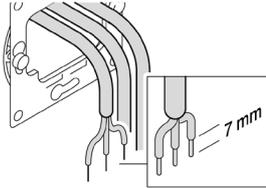
- 3 Strip the cable ends and insulate the cables.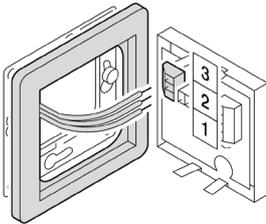


- 4 Feed the cables through the cover frame.
- 5 Connect the device.


- 3: Data_{LB} (room bus II data)
- 2: GND (ground)
- 1: V_{PWR} (supply voltage)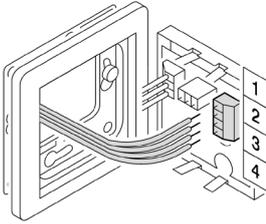


- 6 1: V_{PWR} (supply voltage)
- 2: GND (ground)
- 3: Impulse switch (lamp 1)
- 4: Impulse switch (lamp 2)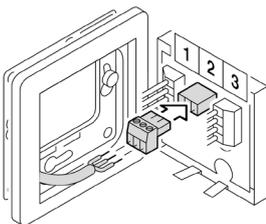

- 7 1: V_{PWR} (supply voltage)
- 2: GND (ground)
- 3: Data_{SB} (system bus data)

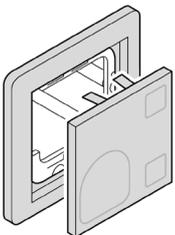
- 8 Position the cover frame and attach the control panel.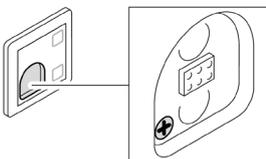


- 9 For devices with IMA socket:
Screw in the screw.


Installing and connecting device with a display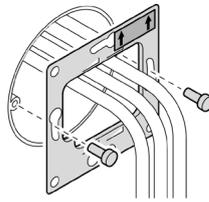

- 1 Feed the cables through the installation frame.
 - 2 Fit the installation frame.
- Important: Type plate sticker top right!

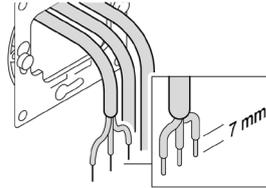
- 3 Strip the cable ends and insulate the cables.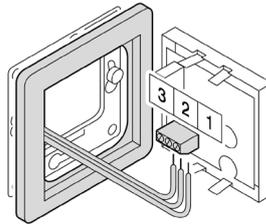


- 4 Feed the cables through the cover frame.
- 5 Connect the device.

- 3: Data_{LB} (room bus II data)
- 2: GND (ground)
- 1: V_{PWR} (supply voltage)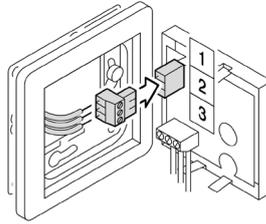

- 6 1: V_{PWR} (supply voltage)
- 2: GND (ground)
- 3: Data_{SB} (system bus data)

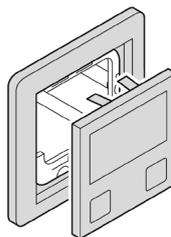- 7 Position the cover frame and attach the control panel.

Checking commissioning requirements

Before commissioning, ensure that the following requirements are met:

- The device is connected to the call system.
- The call system is active.

Only put the device into operation if all commissioning requirements have been met.

Putting the device into operation

Switch on the supply voltage. The device is then ready for use. Feature: The buttons light up (orientation light).

Notify the system administrator so that they can connect the device to the call system from the call centre.

Configuring the device

Configurations are not made on the device.

Notify the system administrator so that they can configure the device from the call centre.

Keeping the device functional

Ensure that the work described in this section is carried out by a "specialist for call systems"

Conducting a quarterly inspection

Check for perfect condition and function:				
▪ Call buttons and movable call triggering devices intended for use by residents or patients	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
▪ Signal lights and acoustic signalling devices	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

Conducting an annual inspection

Check for perfect condition and function:				
▪ All other devices for call triggering, call cancellation and attendance reporting	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
▪ All other display devices	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
▪ All facilities for call enquiry				<input type="radio"/>
▪ All connected devices for receiving, forwarding and displaying calls, e.g. plug-in devices for connecting call-triggering devices		<input type="radio"/>		

Conducting annual maintenance

Carry out the following maintenance work:

- Maintain system components
- Install system updates
- Readjust and calibrate components and devices

Repair

If unauthorised deviations from the set status of the call system are detected:

- Carry out the repair within 24 hours.
- Record the repair in the operating log.

If the call system is completely or partially switched off for maintenance:

- Ensure that the affected rooms are otherwise checked until the call system or the switched-off part is switched on again.

Repairing the device

Safety-relevant device.

Risk of malfunction in the event of improper repair.

Repairs may only be carried out by the manufacturer.

Instructions for operating staff

About the functions of the device

This chapter describes the standard functions of the device. These functions can be changed in the central technology.

If your device behaves differently than described here, ask your system administrator to explain the changed functions.

Using call and cancellation functions

Triggering a call

Press the call button.
The call is triggered.
The red reassurance light lights up.

Canceling the call and setting staff attendance

Press the green attendance button.
The call is cancelled.
The red reassurance light turns off.
Staff attendance is set.
The green reassurance light lights up.

Setting 2nd staff attendance

Press the yellow attendance button.
The 2nd staff attendance is set.
The yellow reassurance light lights up.

Triggering an emergency call

With staff attendance set, also press the call button.
The emergency call is triggered.
The red reassurance light flashes.
The green reassurance light lights up.

Recognising a forwarded call

(only for devices with buzzer)

As soon as you set the staff attendance: The green reassurance light lights up and the buzzer indicates a forwarded call:

- Every 10 seconds: normal call
- Every second: emergency call
- Every half second: alarm call


Deleting a forwarded call

(only for devices with buzzer)

Reset the staff attendance.
The green reassurance light turns off.
The buzzer falls silent.

Recognising an incoming call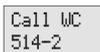

The call tone sounds every 10 seconds.
The display shows the call details.

Line 1: Type of call
(e.g.: call from a WC)
Line 2: Origin of the call
(e.g.: call from room 514, WC 2)

Recognising an incoming emergency call

The call tone sounds every second.
The display shows the emergency call details.

Line 1: Type of call
(e.g.: emergency call from a WC)
Line 2: Origin of the call
(e.g.: emergency call from room 206, WC 1)

Using the remote control

For contactless use of the green attendance/cancellation button:

Hold the NFC transponder in front of the NFC symbol.

Using vital sign call functions

What is a vital sign call function?

The vital sign call function enables the resident or patient to confirm their vitality by pressing the vital sign call button regularly without being prompted.

A missing vital sign report leads to a reminder and, if there is still no vital sign report, to a vital sign call in the system.

What vital sign call functions are available?

- Vitality monitoring:
 - The resident or patient must send a vital sign report at defined intervals^{*)} without being prompted.
 - If the resident or patient does not send the vital sign report, they are reminded visually and acoustically.
 - If the resident or patient has still not responded after the reminder time^{*)} has expired, the system triggers a vital sign call.
- *) The time windows are standardised in the call system management software for all room electronics modules.
- Holiday message:
 - The resident or patient can sign off for their holiday by switching off vitality monitoring.
 - After the holiday, the resident or patient can report back from the holiday by switching the vitality monitoring back on.

Using the vital sign call function: The standard case

Case study: The resident or patient sends a vital sign report unprompted at the defined intervals.

Sending a vital sign report:

Briefly press the vital sign call button unprompted at the defined intervals.

The vital sign report is sent to the call system server.

Responding to a reminder

Case study: The resident or patient has failed to send a vital sign report and is reminded visually and acoustically.

Recognising a reminder:

- The LED flashes at 2-second intervals.
- The buzzer sounds every 30 seconds.
- The system has not yet triggered a vital sign call.

Reacting to the reminder:

Briefly press the vital sign call button.

- The LED turns off.
- The buzzer falls silent.
- The vital sign report is sent to the call system server.

Vital sign call triggered due to lack of response

Case study: The resident or patient did not respond to the reminder, triggering a vital sign call in the system.

Recognising a vital sign call:

- The LED flashes at 2-second intervals.
- The buzzer sounds every 30 seconds.
- The system has triggered a vital sign call because the reminder time has expired.

Note: The triggered vital sign call is only displayed in the call system management software and cannot be distinguished from a reminder on the room electronics module.

Resetting a triggered vital sign call

Case study 1: The resident or patient resets the triggered vital sign call themselves.

Case study 2: The staff visit the resident or patient and reset the vital sign call.

Recognising a vital sign call:

- The LED flashes at 2-second intervals.
- The buzzer sounds every 30 seconds.
- The system has triggered a vital sign call because the reminder time has expired.

Resetting a triggered vital sign call:

Briefly press the vital sign call button.

- The LED turns off.
- The acoustic signal stops.
- The vital sign call is reset.

Signing off for a holiday

Case study: The resident or patient wants to sign off for their holiday and switch off vitality monitoring.

Press the vital sign call button for 3 seconds.

- The buzzer sounds briefly.
- The LED lights up.
- You are signed off for your holiday.
- Vitality monitoring is switched off.

Reporting back from holiday

Case study: The resident or patient wants to report back from holiday and switch the vitality monitoring back on.

Press the vital sign call button for 3 seconds.

- The buzzer sounds briefly.
- The LED turns off.
- You have reported back.
- Vitality monitoring is switched on.

Teaching residents or patients

Explain all the functions that are important to the resident or patient:

- Triggering a call
- Using the vital sign call
 - Sending regular vital sign reports (including information about the time intervals)
 - Responding to a reminder
 - Resetting a triggered vital sign call
 - Signing off for a holiday
 - Reporting back from holiday

Instructions for auxiliary staff

Cleaning

Clean the surfaces with a damp, lint-free cloth. Do not use any aggressive or abrasive cleaning agents.

Disinfecting

Disinfect the surface of the device with a commercially available surface disinfectant according to the instructions.

Heat-sensitive components.
Danger of destruction.
Do not thermally sterilise the device.

Storing the device when not in use

Store the device in its original packaging in a suitable storage location. For the requirements for the storage location, see "Technical data".

Disposing of the device

At the end of the service life:

Observe the regional disposal regulations. Have the device recycled properly. Do not dispose of the device with household waste!